# fläsh.

General instructions for

fläsh Professional Tooth Whitening  
System

Version 1.1

*Handwritten signature: Zimo Walt*

*Handwritten date: 11-06-2021*

**WHITEsmile®**

WhiteSmile GmbH  
Weinheimer Straße 6 · 69488 Birkenau · Germany  
Fon +49(0)6201/84321-90 · Fax +49(0)6201/84321-99  
info@whitesmile.de · www.whitesmile.de

**fläsh Professional Whitening System, including:**

**I. Flash Whitening Lamp (REF 3401), contain:**

Instructions of use page 4-41

- lamp body
- lamp head
- wheels – 5pcs.
- counterweight with screw
- power cable
- exchangeable front parts for lamp head (blue) – 3pcs.
- safety goggles – 3 pcs.
- Instruction for use
- lamp base
- lip retractor (blue)- 3pcs.
- pack of patient brochures
- short instruction A4
- acrylic plate

**II. Kits fläsh (of necessary):**

**1. Kit Flash for 2 patients(REF 3102R):**

Instructions of use page 42-52

- Whitening gel 32% HP (mixed)) 4 g – 2pcs
- Gingiva Protector 1,5g – 2pcs
- After Whitening Mousse White Smile Flash 1,2 ml – 2pcs
- Tips -8 pcs
- Lip retractors– 2pcs
- Whitening Toothpaste White Smile Flash 75ml – 2pcs
- Time-card: 120 min
- Shade Guide

## 2. Kit Flash for 6 patients (REF 3106R):

- Whitening gels 32% HP (mixed)) 4g – 6pcs
- Gingiva Protector 1,5 g – 6pcs
- After Whitening Mousse White Smile Flash 1,2 ml – 6pcs
- Lip retractors – 6pcs
- Tips -24 pcs
- Time-card: 360 min
- Shade Guide

Instructions of use page 42-52

## 3. Kit Flash for 1 patient (REF 3501R):

- Whitening Gel 38% HP (mixed 30%) 2,5 ml
- Gingiva Protector 1,5 g
- After Whitening Mousse White Smile Flash 1,2 ml
- Tips -4 pcs
- Shade Guide

Instructions of us page 54-61

## 4. Kit Power Whitening for 1 patient (REF 5711R):

- Syringe with Power Whitening YF 40% HP (32% mixed) 2,5 ml
- Gingiva Protector 1,5g
- After Whitening Mousse White Smile Flash 1,2 ml
- Tips -4 pcs
- Shade Guide

Instructions of us page 62-67

Use: The system is intended for professional teeth whitening, by means of light and temperature activation of the components of the whitening gel and auxiliary means, due to which a stronger oxygen reaction occurs, providing the best whitening result for the prevention and treatment of dental diseases.

flaesh.com

# fläsh.

## Whitening Lamp

Please follow the instructions of fläsh Whitening lamp!

fläsh.120

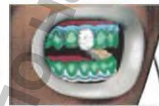
### fläshcard

(Always in the material kit)

(1) THE LAMP CANNOT BE MOVED WITHOUT PULLING THE BLUE BUTTON. (2) FINE ADJUSTMENT CAN BE DONE WITH THE LAMP HEAD BY LOOSEN AND TIGHTEN THE SCREW.

ALL TEETH THAT ARE NOT COVERED WITH GEL MUST BE COVERED WITH MOIST GAUZE.

Restorations must be covered with a moist gauze.


At single tooth whitening all other teeth must be covered with moist gauze.

### Touchscreen Default settings
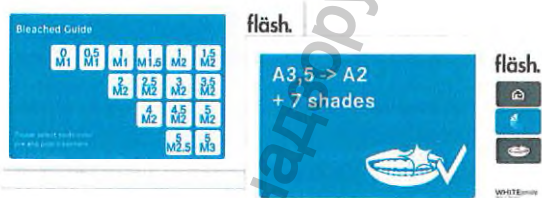
### Determination of achieved tooth shade

### Guided treatment function

**WHITEsmile**  
Made in Germany

WHITEsmile GmbH  
Weinheimer Straße 6  
69488 Birkenau/Germany

Telefon: +49 (0) 62 0 1 / 8 43 21-90  
Fax: +49 (0) 62 0 1 / 8 43 21-99  
E-Mail: [contact@flaesh.co](mailto:contact@flaesh.co)

WHITEsmile #1 in  
Made in Germany

# fläsh.

Whitening Lamp

Instructions for use  
for REF 3401

Version 1.0 - 06/2020

**WHITE**smile  
Made in Germany

[flaesh.com](http://flaesh.com)

Whitening Lamp

6

**Dear Customer,**

we congratulate you to the purchase of the fläsh Whitening Lamp.

These instructions for use are designed to assist you prior to initial use and whenever you require information later on. It is important to observe all safety information to prevent personal injury and material damage. Always keep the operating instructions handy. Please perform maintenance and cleaning based on the corresponding instructions.

We wish you much success and pleasure with the fläsh whitening system.

**Your fläsh Team**

WHITEsmile GmbH, Germany

# fläsh.

## Whitening System

**Latest LED technology**

**Guided treatment function**

**Determination of whitening shades achieved**

**Cosmetic and medical tooth whitening**

To be used only with:

**fläsh Whitening Gel**

With chlorophyll, for effective and gentle treatments.

(not content of the whitening lamp)

**WHITEsmile**  
Made in Germany

[flaesh.com](http://flaesh.com)

## fläsh Whitening Lamp – Instructions for use

### Content

1. General information.....	10
1.1 Contact data .....	10
1.2 Intended Use .....	10
1.3 Indication/Contraindication .....	10
<hr/>	
2. Safety information.....	11
2.1 Standards and regulations.....	11
2.2 Operating personnel .....	11
2.3 Physical working properties .....	11
2.4 Light emission hazards .....	12
2.5 Isolation SUPPLY MAINS .....	13
2.6 Contamination.....	13
2.7 Conditions of use .....	13
2.8 Installation.....	13
2.9 Modifications.....	14
2.10 Whitening procedure: use of whitening gel .....	14
<hr/>	
3. Technical specifications.....	15
<hr/>	
4. Packaging / components .....	16
<hr/>	
5. Installation.....	17
<hr/>	
6. Easy lift system .....	18
<hr/>	
7. Adjusting lamp / storage.....	19

Whitening Lamp

8

8. Operation of user interface .....	20
8.1 Home screen and functions .....	20
8.2 Tooth shade achieved determination .....	22
8.3 Default settings .....	23
8.4 Guided treatment function .....	24
<hr/>	
9. fläshcard time management system .....	26
<hr/>	
10. Signs and symbols .....	27
<hr/>	
11. Whitening procedure - preparation of the whitening lamp .....	28
<hr/>	
12. Maintenance and service .....	30
12.1 Safety checks .....	30
12.2 Maintenance .....	30
12.3 Trouble shooting simple defects .....	31
12.4 Power supply / fuse exchange .....	32
<hr/>	
13. Disinfection and sterilization .....	33
13.1 Surface disinfection .....	33
13.2 Autoclaving of blue front part .....	33
<hr/>	
14. Disposal .....	34
<hr/>	
15. Guidance and manufacturer's declaration .....	35
<hr/>	
16. Electromagnetic emission .....	35
<hr/>	
17. Warranty .....	39
<hr/>	
18. Labels and Label position .....	40

## 1. General Information

### 1.1 Contact data:

WHITEsmile GmbH  
Weinheimer Straße 6  
69488 Birkenau/Germany  
www.flaesh.com  
info@whitesmile.de  
Tel.: +49 (0) 62 01 /8 43 21 90

### 1.2 Intended Use

The fläsh whitening lamp is intended for the light and temperature intensified activation of components of fläsh chairside Light Whitening gel mixed 32 % HP (medical device class IIa) in the dental clinic as an assistive device. Due to the activation a stronger oxygen reaction allows a better whitening result. According to the instructions the lamp does not come directly into contact with the patient – it is placed close to the fläsh lip retractor. The lamp is used by qualified personnel in connection with the fläsh whitening gel for the treatment of abnormally discolored teeth with medical indication. The treatment time is up to 60 minutes. The whitening lamp is a compulsory accessory for the intended use of the fläsh whitening gel. The fläsh whitening lamp is therefore subject to the medical device directive.

The fläsh whitening lamp can also be used with fläsh cosmetic whitening gel for cosmetic tooth whitening in the dental clinic. A use of the lamp without whitening gel is forbidden.

### 1.3 Indication/Contraindication

The fläsh whitening lamp must not be used on patients under 18 years of age. The fläsh Whitening Lamp must only be used with fläsh whitening Gel and indications and other contraindication must follow the instruction of the fläsh Whitening Gel. Local regulations could vary from this instruction for use.

Whitening Lamp

10

## 2. Safety information

### 2.1 Standards and regulations

The fläsh whitening lamp is manufactured in compliance with the provisions of Council Directive 93/42/EEC (MDD) concerning medical devices.

National directives regarding electrical installations must be observed.

---

### 2.2 Operating personnel

The fläsh Whitening Lamp may only be operated by educated and qualified personnel (dentist, assistant, dental hygienist). The applicable occupational safety regulations and accident prevention measures, the current operating instructions and national requirements concerning education must be complied with.

Users are obliged to use only faultless materials, to ensure correct application and to protect themselves, the patient and other persons against hazards.

In order to prevent false or improper use, the fläsh Whitening Lamp must not be used by unauthorized persons. Therefore the fläsh Whitening Lamp must be protected against unauthorized access when not in use.

---

### 2.3 Physical working principle

The blue light with 465 nm of the fläsh Whitening Lamp is generated by 3 LED's. The power of the LED's can be adjusted. The light and the developed temperature in the whitening gel accelerate the oxygen reaction allowing better whitening results.

## 2.4 Light emission hazards

The flash Whitening lamp emits blue light with a light spectrum of 460 nm. The maximum output of the light is 190 mW/cm<sup>2</sup> in CONSTANT Mode and power setting III 100%. The output can be lowered: III 75 % - II 50 % - I 25 %.

The minimum distance to the skin is 3 cm. It is technically determined by the blue silicon part on the lamp head and the lip retractor and will always be more than 3 cm to the gingiva of the patient. Additional skin protection is not necessary.

Do not stare into the light source on the lamp head. Make sure that patient and operator are wearing orange safety goggles according DIN EN 166:2002-04 which are delivered with the flash whitening lamp when LED's are activated.

The light can be stopped with the PAUSE or STOP button on the screen or with the black main switch below the lamp body close to the power cable.

The plexi glass in front of the LED's must be unblocked in order to prevent heat development - do not stick anything on the glass and make sure it is clear before each time of use.

**CAUTION** - Use of controls or adjustments or performance of procedures other than those specified herein may result in hazardous radiation exposure.

Light output

Whitening Lamp

12

## 2.5 Isolation from supply mains

The fläsh whitening lamp can be disconnected from supply mains electrical power by pulling the power cable out of the socket. Do not position the lamp in a way that it is difficult to disconnect the device.

## 2.6 Contamination

The fläsh whitening lamp is only touching the lip retractor with the exchangeable blue front part of the lamp head. In order to prevent the danger of (cross) contamination autoclave the front part before each time of use the fläsh Whi ening Lamp following the autoclaving instruction in chapter 13.

Always use disinfected blue front part for each patient.  
Do not use the fläsh Whitening Lamp without blue front part.

## 2.7 Conditions of use

The fläsh whitening lamp can only be used in the dental clinics, hospital environment or in dental studios. (or if local regulation allow the cosmetic whitening also in whitening centers). The use can only be performed indoors at a temperature between 10 °C to 35 °C. The EMC regulations must be considered: mobile RF communication equipment can effect medical device equipment. The use is only allowed with the power cable delivered by WHITEsmile.

## 2.8 Installation

The fläsh whitening lamp must be set up properly and completely before putting it into operation, see chapter 5 „Installation“. Do not use the fläsh Whitening Lamp if a visual inspection shows that it has been damaged.

## 2.9 Modifications

Modifications of this system which might affect the safety of the system owner, patients or other persons are prohibited by law! The user is responsible for any damage resulting from the use of nonapproved accessories. It is not permitted to modify the design or construction of the unit.

The operator is allowed to change the fuse and power cord as explained in Chapter 12.4. The user is also allowed to exchange the head with spare part REF 3410.

All other repair must be done with authorized personal. Please ask WHITEsmile or your local dealer where you purchased the flash whitening lamp.

## 2.10 Whitening procedure: use of whitening gels

For reasons of product safety, this product may be operated only with original WHITEsmile flash Gels in order to prevent uncontrolled heat development. The whitening procedure must follow the instructions of the flash whitening gels which always have the most up to date information about time and settings.

**CAUTION:** All teeth that should not be bleached and are therefore not covered with gel, must be covered with moist gauze in order to prevent heating (to be covered have to be restorations like crowns, bridges, implant suprastructures and veneers or in case of single tooth whitening the other teeth). Gingiva protector can be used for better fixation.

Restorations must be covered with a moist gauze.

At single tooth whitening all other teeth must be covered with moist gauze.

Whitening Lamp

14

### 3. Technical specifications

fläsh Whitening Lamp is an electronic unit that controls 3 blue LED (Light Emitting Diodes). This unit is used to optimize dental tooth whitening

#### Technical Information

Power supply	100-240V ~ 50 Hz-36 VA
Max input power	60 W
Time-delay fuse	T1.6A H, 250 V
Light spectrum	460 nm
High-intensity lighting source by blue led	3 LED
Protection against electric shock	Class II
Operating Mode	Time on 20 min, Time off 1 in, no continuous use.

#### Operating: environmental conditions

Ambient Temperature	10°C to 35°C (50°F - 95°F)
Relative humidity	5% to 95% not including condensation
Atmospheric pressure	70 kPa to 106 kPa
Max. Altitude	2000m

#### Transport & Storage: environmental conditions

Ambient Temperature	-40°C to 70°C (-40°F - 158°F)
Relative humidity	5% to 95% not including condensation
Atmospheric pressure	50 kPa to 106 kPa

#### Physical Specifications

Power cord plug	SCHUKO
Power cord length	3,00 m
Dimensions	152 × 120 × 63 cm (max. dimensions of build up lamp) 63 x 140 x 63 cm, folded
Total mass assembled lamp	7,5 kg

**CE** Medical device CE 93/42: CLASS I  
complies with IEC/EN 60601-1

WHITEsmile GmbH  
Weinheimer Strasse 6  
69488 Birkenau/Germany  
info@whitesmile.de  
Tel.: +49 (0) 62 01 /8 43 21 90  
MADE IN GERMANY

Fuse T 1.6A H, 250 V  
100-240 V ~ 50-60 Hz 36 VA

The expected service life of the fläsh whitening lamp is 10 years.

#### NOTE FOR SERVICE PERSONNEL:

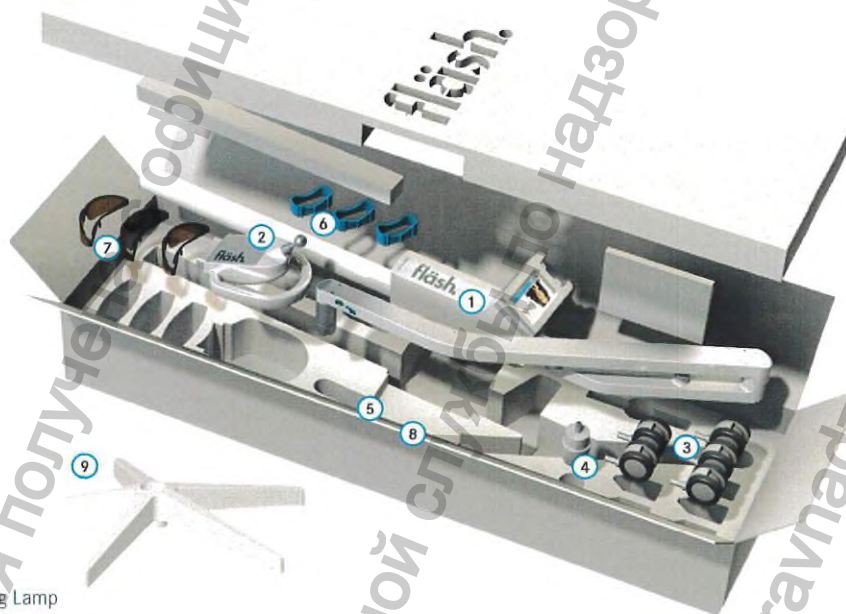
A service manual for authorized service personnel is available from WHITEsmile for technicians and dental dealers. Information like circuit diagrams, component part lists, descriptions are available upon request. Please contact info@whitesmile.de.

#### 4. Packaging / components

**Content main carton:** ① 1 lamp body / ② 1 lamp head / ③ 5 wheels / ④ 1 counterweight with screw  
⑤ 1 power cable / ⑥ 3 exchangeable front parts for lamp head (blue) / ⑦ 3 safety goggles according to  
DIN EN 166: 2002-04 (REF 3470). / ⑧ Instruction for use / **Dimensions carton 1:** 142 x 40 x 24 cm, 10 kg.

**Packed separately:** ⑨ 1 lamp base. **Dimensions carton 2:** 62 x 62 x 15 cm, 4 kg.

Use only original power cable provided with the fläsh Whi ening Lamp.

Whitening Lamp  
16


## 5. Installation
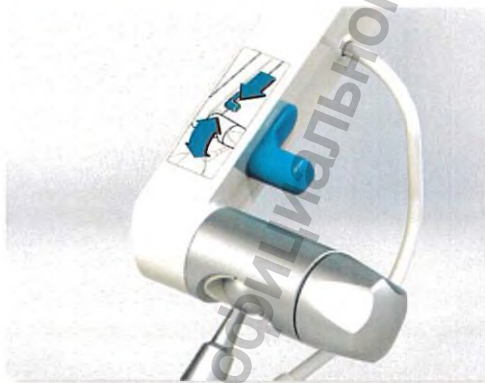
## 6. Easy lift system


The fläsh Whitening Lamp has a unique positioning system. The blue handle (1) releases the lamp to an easy positioning in front of the patient on a 3D level. Once position is found the button is released and the lamp stays put exactly where it should.

**PLEASE NOTE: THE LAMP CANNOT BE ADJUSTED WITHOUT THE BLUE BUTTON!**

Whitening Lamp

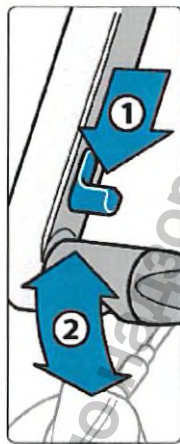18

(2) Fine adjustments can be done with the lamp head by loosening and tightening the screw.

## 7. Adjusting lamp/storage

The fläsh Whitening Lamp offers a lot of possibilities to adjust the head of the lamp in front of the patient's mouth. The head can be moved in 3 dimensions. Additionally, the angle of the holding arm of the head can be adjusted.

As shown aside, the lamp can be arranged in front of the patient's position, from the side and even from the back.

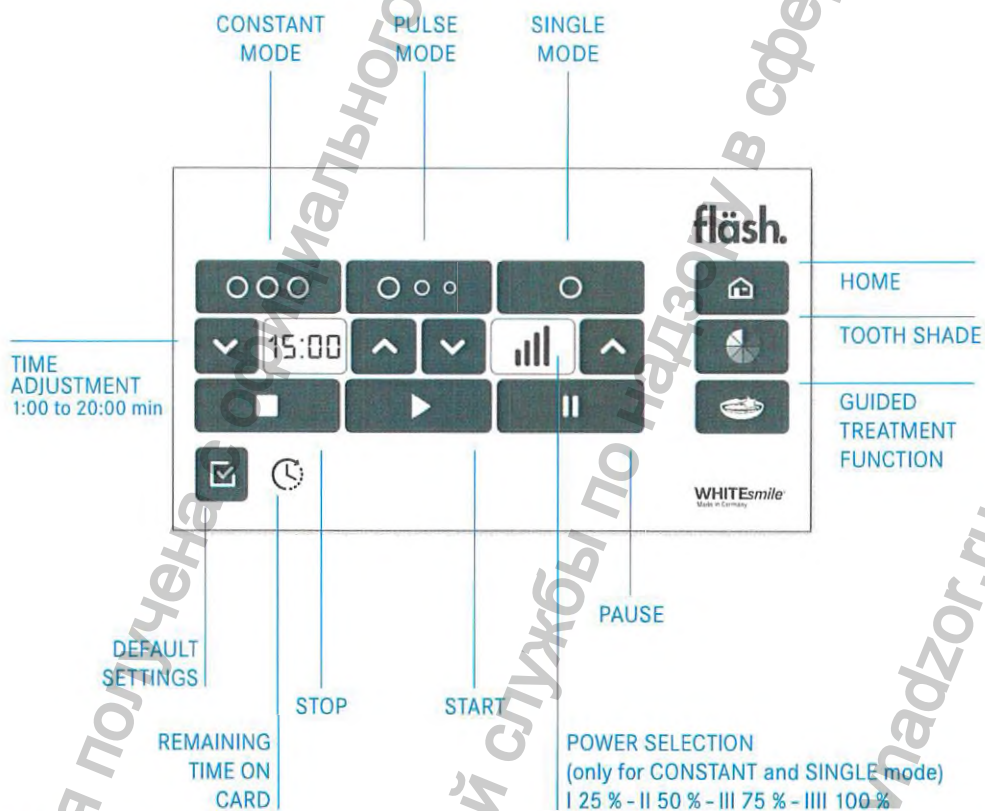
For storage or transportation, for example: in other rooms use cable roll-up on back side of the fläsh whitening lamp to avoid stepping on the cable.

Total weight of assembled lamp: 7,5 kg.

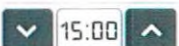
## 8. Operation of user interface

### 8.1 Home screen and functions

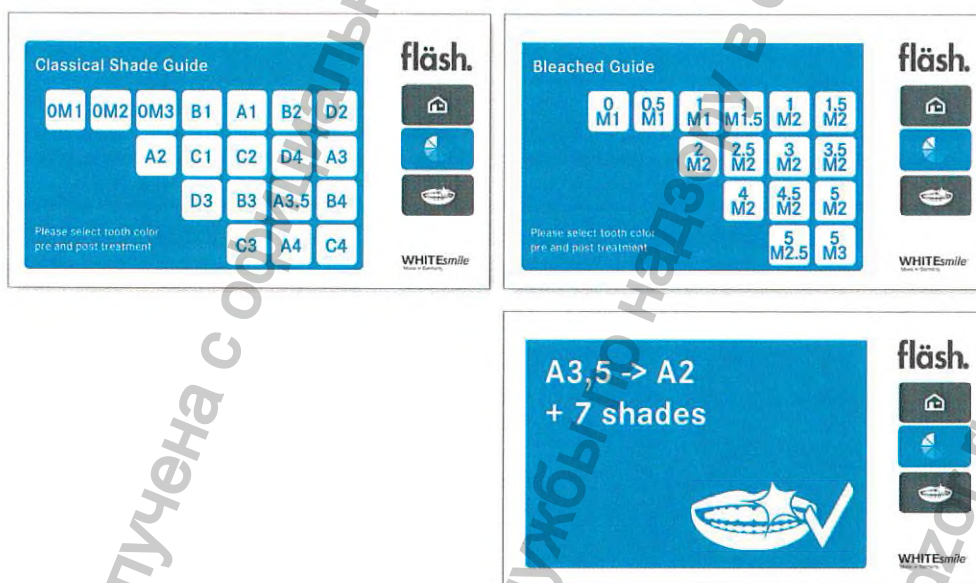
Whitening Lamp  
20

	<b>CONSTANT MODE</b>	All 3 LED's are constantly working. Light intensity can be adjusted in 4 levels.
	<b>PULSE MODE</b>	The 3 LED's are activated in a sequence. This mode has a high light intensity with low temperature increase. It is recommended for sensitive patients.
	<b>SINGLE MODE</b>	For single tooth whitening - only middle LED is on. Light intensity can be adjusted in 4 levels. Does not use time credits from the flashcard. A card must be in the slot to operate. Do not use single mode for full whitening. It will result in an uneven whitening result. Can be adjusted from 1:00 to 20:00 minutes.
	<b>TIME ADJUSTMENT</b>	
	<b>POWER ADJUSTMENT</b>	For CONSTANT and SINGLE mode the light intensity can be adjusted to 25 % / 50 % / 75 % / 100 %. The higher the level the more temperature increase in the gel.
	<b>START</b>	Pressing the "START" button the procedure begins and the LED's go on.
	<b>PAUSE</b>	Pressing the "PAUSE" button holds the process and the time stays on the current value.
	<b>STOP</b>	Pressing the "STOP" button ends the procedure and the lamp jumps back in the original setting.
	<b>TOOTH SHADE</b>	Determines the whitening shade achieved when pre and post treatment shade is entered. Works with classical or bleached guide.
	<b>GUIDED TREATMENT FUNCTION</b>	Allows step by step guided treatments for a perfect result.
	<b>DEFAULT SETTINGS</b>	Allows to preset values for minutes per cycle, power level, kind of shade guide used and language selection.
	<b>REMAINING TIME ON CARD</b>	Shows the remaining time on flashcard. Counts down if PULSE or CONSTANT program are used by the minute. If card is not inserted or empty light on lamp can not be started.

## 8.2 Tooth shade achieved determination

Please select the shade guide you want to use in the default settings. You can use the VITA classical shade guide or the VITA Bleached Guide\*. Determine the pre and post treatment shade and select it on the screen. By touching the post treatment shades the fläsh Whitening Lamp immediately shows the achieved tooth shade improvement.


\*Registered trademark Vita Zahnfabrik, Bad Säckingen, Germany.

Whitening Lamp  
22

### 8.3 Default settings

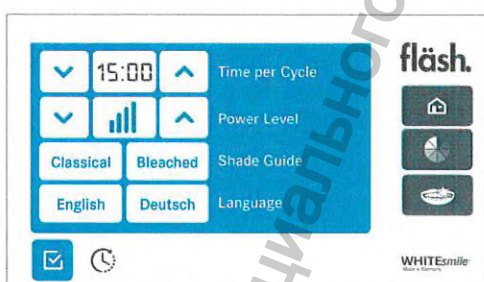The changes in the default settings are stored in the lamp memory until changed again.


## 8.4 Guided treatment function

The unique guided treatment function allows to go through the whole whitening procedure step by step with illustrated user instructions. It allows a standard procedure every time. The function can be returned to a normal operation by pressing the home button.

There is also a step-by-step video available at [www.flaesh.com](http://www.flaesh.com)



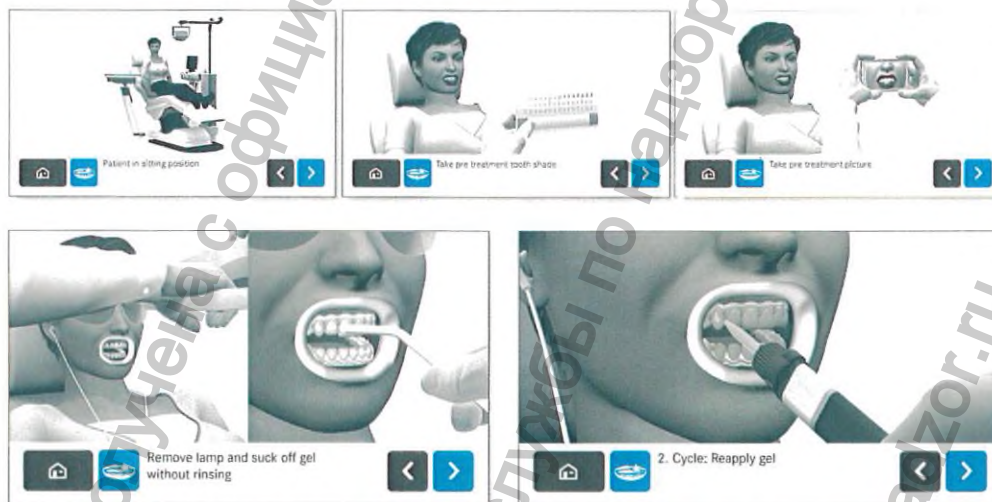
**NOTE:** the operator has to keep track of the treatment as well. If the lamp is turned off during treatment the guided treatment function will start from the beginning again. Or if treatment steps are not done the lamp will not recognize.

Whitening Lamp

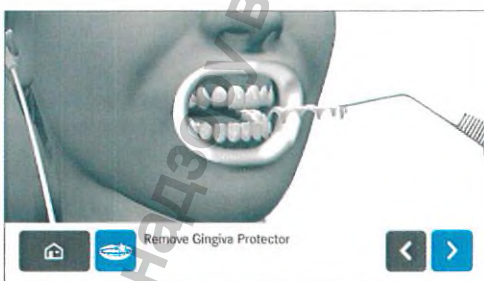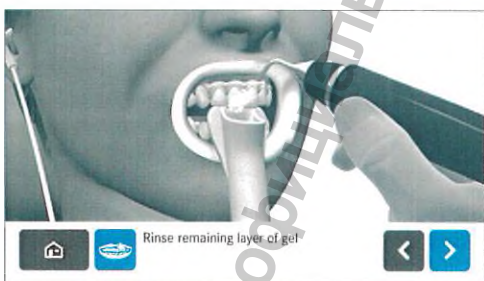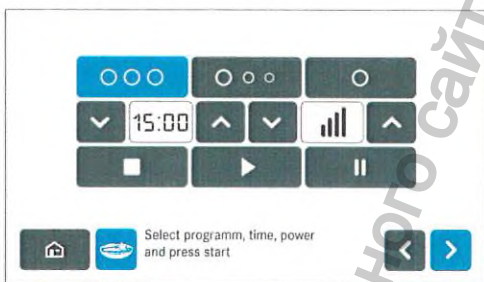24

## 9. fläshcard time management system

360 minutes card in 6 patient kit.  
120 minutes card in 2 patient kit.

### > Replacement card in case of missing card <<

In each fläsh material kit for each patient 60 minutes treatment time are provided. The card can be conveniently placed into the lamp. Remaining time is stored on the card and can be used also on another fläsh Whitening Lamp.

SINGLE MODE for single tooth whitening can be used without using the time on the card. Please note that also SINGLE MODE requires a card to be in the slot.

**In the single mode only the middle LED is on - do not use for full whitening sessions – the result will be uneven in front and side.**

Whitening Lamp

26

## 10. Signs and symbols flash lamps

	CE-Symbol		Special recycling (WEEE)
	Serial number	<b>REF</b>	Article number
	Read user manual!		Warning for optical radiation aperture
	Manufacturer		Protection against electric shock class II
	Switching on/off		For indoor use only
	Year of production		
	CAUTION		

## 11. Whitening procedure – preparation of the whitening lamp

1. Perform safety check as described in chapter 12.1.
2. While LED's are on do not stare at the light source. A warning sticker is located on the lamp head.
3. See instruction of WHITEsmile fläsh whitening gel for whitening procedure and for recommended treatment cycles.

Take in consideration the following information:

Place lamp in front of the patient slightly touching the lip retractor.

**CAUTION:** Use on teeth not covered with gel might lead to higher temperature on the tooth surface.

**NOTE:** fläsh Light Whitening Gel becomes clear after a few minutes, showing the strong activation of the fläsh Whitening Lamp. The gel is still active and remains on the teeth until the end of the cycle and the lamp goes off

Restorations must be covered with a moist gauze.

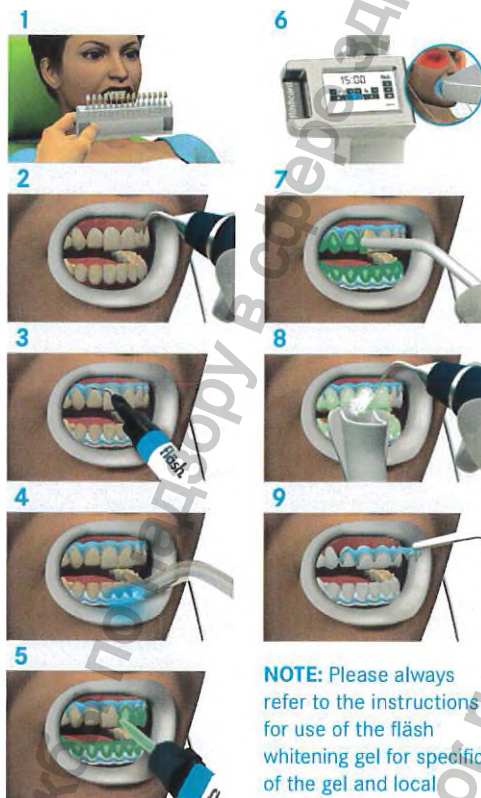
At single tooth whitening all other teeth must be covered with moist gauze.

Whitening Lamp

28

Whitening procedure  
scheme:

**NOTE:** Please always refer to the instructions for use of the flash whitening gel for specific of the gel and local requirements.

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## 12. Maintenance and service

### 12.1 Safety checks

Before each use of the fläsh whitening lamps the following safety checks must be performed by the user:

1. Make sure that the surface of the fläsh Whitening Lamp has been wipe disinfected after last use.
2. Make sure that a fresh blue silicon part (autoclavable) is located on the light output of the lamp head.
3. Check if the glass in front of the LED's on the lamp head is clean and the LED's are clearly visible. Do not put any tape or other material on the glass in front of the LED.
4. Check if all 3 LED's are on: wear safety goggles and turn on lamp in constant mode. If not all 3 LED's are on - do not use the lamp.

There is no general safety check by a service engineer required. If national or local legal regulations require additional safety checks for your fläsh Whi ening Lamp, these regulations must be complied with and the corresponding checks must be performed.

### 12.2 Maintenance

The fläsh Whitening Lamp does not require special maintenance. Due to the high expected working time of the LED's there is no change in the power of the LED's to be expected in the life span of the fläsh Whitening Lamp.

The manufacturer accepts responsibility for the safety of the fläsh Whitening Lamp only if the following requirements are fulfilled:

Modifications of he fläsh Whitening Lamp or repair work may be performed only by authorized personnel.

The electrical installations in the rooms where the fläsh Whitening Lamp is used must fulfill he applicable legal requirements.

The fläsh Whitening Lamp must be used in compliance with the instructions provided in the present manual.

Whitening Lamp

30

### 12.3 Troubleshooting simple defects

In case of malfunctioning, proceed as follows:

Even by inserting a flashcard (as described in chapter 9) in the slot - working with the flash Whitening Lamp is not possible: try other time card from another whitening kit or use spare card provided in this instructions of use. If the problem persists, contact your dealer.

Touch screen is black - turn machine off. Check if all cables are connected correctly.

Check if power switch is on. If not, check if plug has electricity and change fuse as described in chapter 12.4. If problem persist contact your dealer.

If power switch is on and touch screen black - please call your dealer.

Touch screen is on and procedure can be started, but LED's do not start: check connection of lamp head. If problem persist contact your dealer.

If not all LED's are on even at constant mode: contact your dealer.

---

#### NOTE FOR SERVICE PERSONNEL:

A service manual for authorized service personnel is available from WHITESmile for technicians and dental dealers. Information like circuit diagrams, component part lists, descriptions are available upon request. Please contact [info@whitesmile.de](mailto:info@whitesmile.de).

## 12.4 Power supply / fuse exchange

Double power supply switch with signals „I” for ON and „0” for OFF.  
In case of an emergency stop, press off and remove power plug to disconnect the lamp from power source.

100-240 V ~ 50 Hz-60 Hz VDE type float socket.

The power cord is plugged into the socket below the user interface box. For safe disconnection, always unplug the flashing lamp from the power outlet when is not used. Always use 2 fuses with specification: T 1.6A H, 250

### Fuse replacement instruction (can be done by the user):

1. Unplug power plug.

2. Pull fuse box out. If difficult with help of small screw driver.
3. Take out 2 used fuses and replace by 2 new fuses:  
spare fuses are being delivered with flashing lamp.  
Otherwise use fuse following specification
4. Reinsert fuse box.
5. Reinsert power plug.
6. Turn on lamp.

Always use original delivered power cord or spare part by WHITEsmile.

Whitening Lamp

32

## 13. Disinfection

### 13.1 Surface disinfection

Before and after using the device disinfection measurements are considered. Before cleaning the device, switch it off and unplug the lamp from the electric power outlet. The device can be cleaned and disinfected with by wipe disinfection a soft cloth slightly moistened with nontoxic and non-flammable solvents: tested disinfection solvents for plastic surfaces available on the market are recommended. Avoid the entry of liquid into device components. Do not use strong solvents.

### 13.2 Autoclaving of blue front part

In order to avoid cross contamination from patient to patient disinfect the blue detachable front part of the lamp head in the autoclave by 134°C for 3 minutes following the instructions of the autoclave manufacturer. The part does not have to be sealed and sterile storage is not required.

There are 3 blue parts delivered with each lamp. The blue front part can be autoclaved for 100 times. A pack of 3 pieces can be reordered with REF 3497.

## 14. Disposal

The expected service life of the fläsh whitening lamp is 10 years.

Based on Directive 2012/19/EU and country-specific disposal directives for waste electrical and electronic equipment, we would like to point out that these must be disposed of in a special way within the European Union (EU). These regulations require environmentally compliant recycling/disposal of waste electrical and electronic equipment. They must not be disposed of as domestic waste. This is shown with the symbol of the crossed out dust bin, which has been in use since March 24, 2006.

For disposal procedure please contact your local dealer.

Whitening Lamp  
34

### Other countries

The dental dealers would be glad to provide you with country-specific information.

## 15. Guidance and manufacturer's declaration

The fläsh Whitening Lamp is a medical electrical equipment that needs special precautions regarding EMC and needs to be installed according to EMC information below: EN ISO IEC 60601-1-2

**WARNING:** Mobile RF communications equipment can affect the fläsh Whitening Lamp. Usage of other cables or accessories than delivered with the fläsh Whitening Lamp might affect the EMC performance.

The fläsh Whitening Lamp should not be placed close to other equipment.

All relevant test for EMC have been performed under working conditions of intended use

## 16. Electromagnetic emission

The fläsh Whitening Lamp is intended for operation in the electromagnetic environment specified below. The customer or user of the fläsh Whitening Lamp should make sure that it is used in such an environment.

Emission measurement	Conformity	Electromagnetic environment – guidelines
RF emissions according to CISPR 11	Group 1 <sup>a</sup>	The fläsh whitening lamp uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions according to CISPR 11	Class B	The fläsh whitening lamp is intended for use in all facilities, including residential areas and those directly connected to a public power supply, which also provides electricity to buildings used for residential purposes.
Harmonics according to IEC 61000-3-2	Class A	
Voltage fluctuations/flicker according to IEC 61000-3-3	Complies	

## Interference immunity

The flash whitening lamp is intended for operation in the electromagnetic environment specified below. The customer or user of the flash whitening lamp should make sure that it is used in such an environment.

Interference immunity tests	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidelines
Electrostatic discharge (ESD) according to IEC 61000-4-2	± 8 kV contact discharge ± 15 kV air discharge	± 8 kV contact discharge ± 15 kV air discharge	Floors should be made of wood or concrete or finished with ceramic tiling. If the floor is covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst according to IEC 61000-4-4	± 1 kV for input and output lines ± 2 kV for power supply lines	± 1 kV for input and output lines ± 2 kV for power supply lines	The quality of the line power supply should be that of a typical commercial or hospital environment.
Surge voltages according to IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode voltage	± 1 kV differential mode ± 2 kV common mode voltage	The quality of the line power supply should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and variations of the power supply according to IEC 61000-4-11	<5% $U_T$ for ½ period (>95% dip of $U_T$ ) 40% $U_T$ for 5 periods (60% dip of $U_T$ ) 70% $U_T$ for 25 periods (30% dip of $U_T$ ) <5% $U_T$ for 5sec. (>95% dip of $U_T$ )	<5% $U_T$ for ½ period (>95% dip of $U_T$ ) 40% $U_T$ for 5 periods (60% dip of $U_T$ ) 70% $U_T$ for 25 periods (30% dip of $U_T$ ) <5% $U_T$ for 5sec. (>95% dip of $U_T$ )	The quality of the line power supply should be that of a typical commercial or hospital environment.  If the user of the flash whitening lamp requires it to continue functioning following interruptions of the power supply, it is recommended to have the flash whitening lamp powered by an uninterruptible power supply or a battery.

Whitening Lamp

36

Interference immunity tests	IEC 60601-1-2 test level	Compliance level	Electromagnetic environment - guidelines
Magnetic field of power frequencies (50/60 Hz) according to IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note: $U_T$ is the AC supply voltage prior to application of the test level.			
			Portable and mobile radio equipment must not be used within the recommended working clearance from the flash whitening lamp and its cables, which is calculated based on the equation suitable for the relevant transmission frequency. Recommended working clearance:
Conducted RF disturbance IEC 61000-4-6	$3 V_{\text{eff}}$ 150 kHz to 80 MHz <sup>1</sup>	$3 V_{\text{eff}}$	$d = [1, 2] \sqrt{P}$
Radiated RF Interference IEC 61000-4-3 <sup>4</sup>	$3 V/m$ 80 MHz - 800 MHz <sup>1</sup> $3 V/m$ 800 MHz - 2.7 GHz <sup>1</sup>	$3 V_{\text{eff}}$  $3 V_{\text{eff}}$	$d = [1, 2] \sqrt{P}$ at 80 MHz - 800 MHz $d = [2, 3] \sqrt{P}$ at 800 MHz - 2.7 GHz with $P$ as the power rating of the transmitter in watts (W) according to the transmitter manufacturer's specifications and $d$ as recommended safety distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey <sup>2</sup> should be less than the compliance level <sup>3</sup> in each frequency range. interference is possible in the vicinity of equipment bearing the following  graphic symbol.

## Working clearances

**Recommended working clearances between portable and mobile HF communication devices and the fläsh whitening lamp.**

The fläsh whitening lamp is intended for operation in an electromagnetic environment, where radiated RF interference is checked. The customer or the user of the fläsh whitening lamp can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the fläsh whitening lamp - depending on the maximum output power of the communication device, as shown below.

Power rating of the transmitter [W]	Working clearance according to transmission frequency [m]		
	150 kHz - 80 MHz	80 MHz - 800 MHz	800 MHz - 2.5 GHz
	$d = [1,2] \sqrt{P}$	$d = [1,2] \sqrt{P}$	$d = [2,3] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

Whitening Lamp

38

## 17. Warranty

WHITEsmile or your local dealer will repair or at its option replace, without charge, your product which proves to be defective in material or workmanship under normal use, during the warranty period of 2 years from invoice date. Freight cost and compensation for absent device are not covered by the warranty. The above warranty does not cover damages or problems which result from misuse, abuse, accident, alteration, or incorrect use not in accordance with the manufacturer's instructions.

WHITEsmile authorizes the repairing of its devices to those laboratories with qualified personnel (electrical engineers) who shall receive the necessary information for repairing and testing the product.

The testing equipment and spare parts shall be sent by request. Note: the repairing operation must follow the actual required standards.

In case of breakdown of the apparatus contact your local dealer or:

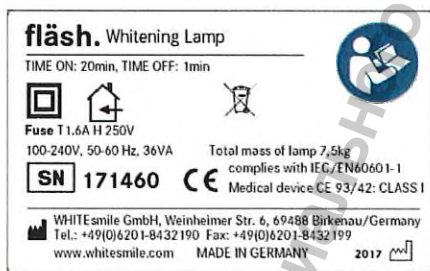
**WHITEsmile GmbH**  
**Weinheimer Strasse 6**  
**69488 Birkenau/Germany**  
**Phone:+49 (0) 62 01 / 8 43 21 90**  
**E-Mail: info@whitesmile.de**

No compensation can be demanded from the purchaser for the detention of the equipment.

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Whitening Lamp  
40

Labels on lamp head:

CAUTION: The light emitted may be harmful to the eyes.  
Do not stare at the light source.

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# fläsh.

## Whitening Material

For each patient the kits all necessary components for the whitening procedure:

- Whitening Gel 4g Gel 32% HP (mixed) - (packed separately)
- 1,5 g Gingiva Protector
- AWM 1,2 ml
- Lip retractor
- 60 min per patient

### 1) Preparation

use from kit:  
fläsh Time card (60 min per patient)  
Lip retractor

**Gingiva Protector**  
use from kit:  
**Gingiva Protector**

**Treatment**  
use from kit:  
**Whitening Gel**

Repeat treatment 3 x. (max. 4 x)

**Postprocessing**  
use from kit:  
**After Whitening Mousse (AWM)**

# fläsh.

## Whitening Material

### Product description

fläsh whitening materials are intended to whiten teeth in a medical indicated way. The indication of use can be found for each individual in the instruction chapter of this manual.

1. Qualification of the user: dental staff or dentist – basic knowledge of whitening by instruction
2. User profile: The intended user profile is dentists or dental professional qualified for tooth whitening treatments. Due to the fact that the user will have to do the whitening treatment, which requires knowledge about prophylaxis and handling a procedure in the patient mouth, there is no additional qualification needed to use the fläsh Whitening Lamp. A study of the instructions of use of course is mandatory for the user.
3. Patient Profile: The fläsh Whitening Gels can be used in conjunction with the fläsh Whitening Lamp. All specification about the patients (i.e. older than 18 years of age, not pregnant or lactating, good oral health etc.) that can be treated are mentioned in the instructions of the flash chairside whitening Gel 32% HP mixed\* (medical tooth whitening).
4. Storage/Transport: Material must be stored reffridgerated 4-12°C. Temperature at transport must not exceed 40 °C for not longer than 10 days.

5. Content of flash Light activated kits

Whitening Material: Gel 4g Gel 32% HP (mixed) – (packed separately) Gingival  
Barrier to protect soft tissue: 1,5 g Gingiva Protector  
After Whitening Mousse desensitizer: AWM 1,2 ml  
Autoclavable lip retractor: Lip retractor  
Time-card to activate lamp and guarantee that flash gel is used: 60 min per patient

6. Indication of use - see individual instruction for each gel in this manual.

7. Contraindications - see individual instruction for each gel in this manual.

8. Side effects - see individual instruction for each gel in this manual.

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## Kits for use with the flash Whitening lamp:

### 1. 2 Patient Kit (REF 3102R)

- 4g Gel 32% HP (mixed) – 2 pcs. (packed separately)
- 1,5 g Gingiva Protector- 2pcs.
- AWM 1,2 ml – 2pcs.
- Lip retractor – 2pcs.
- Toothpaste 75 ml – 2pcs.,
- Time-card 120 min
- shadeguide
- 8 tips

### 2. 6 Patient Kit (REF 3106R)

- 4g Gel 32% HP (mixed) –6 pcs. (packed separately)
- 1,5 g Gingiva Protector- 6pcs.
- AWM 1,2 ml – 6pcs.
- Lip retractor – 6pcs.
- Time-card 360 min
- shadeguide
- Tips - 24 pcs

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Instructions for REF 3102R, 3106R,  
on following pages =>

# fläsh.

Chairside Light  
Whitening System

Information for the Dentist  
Medical Tooth Whitening

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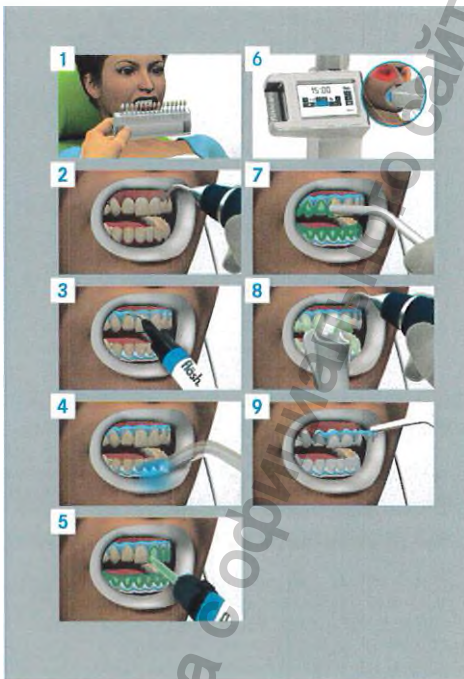
#### WHITEmile fläsh Chairside Light Whitening System

**Medical Tooth Whitening System for chair side use in the dental clinic for discolored vital and non-vital teeth, only with medical indications. Warning: Keep out of the reach of children. According to the Medical Device Directive 93/42/EEC.**

**IMPORTANT NOTE:** fläsh Chairside Light Whitening System is unique in technique and formulation. Please carefully read and understand the instructions before proceeding with the whitening treatment. fläsh Chairside Light Whitening Gel is a 2-component material for whitening discolored teeth. It is intended only for professional medical whitening in the dental office. By mixing the two components of the bleaching agent the gel gets activated. The duration of the treatment is determined individually and depends on the sensitivity of the teeth, to the desired degree of whitening and the nature and extent of the tooth discolorations.

fläsh Chairside Light Whitening Gel is available in a mixed concentration of 32 % Hydrogen Peroxide for the use with the fläsh Whitening Lamp.

The green colored gel can be applied directly from the mixing tip to the teeth - its viscous consistency prevents accidental dripping or flow. Unused gel can be kept in the closed syringe until the end of the shelf-life. It can be used with a new mixing tip for another treatment.

ENGLISH

Chairside Light Whitening System

48

#### Indication for use

fläsh Chairside Light Whitening System is used for fast, medically indicated whitening of strongly discolored, vital and non-vital teeth, which cannot be converted by a professional dental tooth cleaning in the natural or normal tooth color. Medical tooth whitening is indicated when discolorations lead to a deterioration of (health-related) quality of life of the patient because of their abnormally deviating tooth color. Medical tooth whitening can also be used as an alternative to (significantly more invasive) conservative treatment of the affected teeth with veneers or crowns.

Natural or normal are tooth colors as they usually occur in their brightness, translucency and color in the population. For diagnosis and indication for example the A1-D4 Vita classical shade guide can be used (for the most frequently occurring tooth colors).

Examples of the causes of such (intrinsic) discoloration are: tetracycline/minocyclin discolorations, moderate to severe dental fluorosis (TF index > 2), non-vital teeth (pulp necrosis) - e.g. by trauma, genetic or age-related (degenerative) discoloration, porphyria, or jaundice, heavy deposits of foods with penetration into the tooth substance (intrinsic discoloration by extrinsic dyes). It can be used to whiten single or multiple teeth or parts of teeth. It can also be used for whitening of single endodontically treated non-vital teeth including intracoronary bleaching.

#### Warning

Because of the high concentration of Hydrogen Peroxide fläsh Chairside Light Whitening Gel is highly reactive. To avoid contact with skin or eyes we recommend the use of gloves and safety goggles. For safety reasons, the patient should also wear safety goggles. If the gel comes into contact with skin, remove it and rinse off with water immediately.

White spots may appear in the affected area and the patient may experience a burning sensation.

The symptoms should disappear after approx. 30 - 60 minutes. If the gel comes into contact with the eyes, rinse immediately and consult a doctor.

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### 1) Preparation

- **Before the whitening treatment a diagnostic indication for medical indicated tooth whitening has to be defined by the dentist.** After examining the patient the current tooth color should be determined and documented by the dentist. A professional tooth cleaning is indicated prior (ideally 1-2 weeks) to the whitening treatment. Important: due to the fact that restorative materials cannot be whitened, it is recommended that teeth be whitened prior to prosthetic restorations. Any prosthetic treatment (crowns, fillings etc.) should be done 1 - 2 weeks after the whitening treatment in order to match the color of the whitened teeth.
- Please note: in case of single tooth whitening all other teeth that are not being whitened, must be covered with moist gauze. Any prosthetic restorations (such as e.g. crowns) need to be covered with moist gauze as well.
- Start the whitening procedure by applying a thin layer of Vitamin E oil on the patient's lips. Place the flash lip retractor in the patient's mouth. If necessary, use cotton rolls for optimal isolation.

### 2) Application of flash Gingiva Protector

- flash Gingiva Protector is an easy to apply light-curing resin, which allows optimal covering of soft tissue and neighbouring teeth during the whitening treatment. It is also suitable to cover exposed dentin (e.g. tooth necks) and filling materials during the treatment.
- Attach the dispensing tip to the top of the syringe of the flash Gingiva Protector.

- Dry the gingiva carefully to ensure that the flash Gingiva Protector will hold and seal during the complete treatment. Place the flash Gingiva Protector directly on the gingiva and cover 3 - 4 mm of the gingiva and 0.5 - 1 mm of the teeth (use rule "no pink"). Work in quadrants: light cure the gingiva protector after each half jaw finished. Light cure in an arching movement for about 25 seconds. When finished, use dental mirror to check if the seal is tight and no reddish gingiva is visible. Close open spots again with Gingiva Protector and light cure.  
Instead of the flash Gingiva Protector a rubber dam can also be used.
- The patient should be positioned in a slightly declined position (45° degree angle) during the whitening process to be able to swallow comfortable while closing the teeth with light pressure on the flash lip retractor.

### 3) Whitening procedure

- The gel should be used at room temperature. Attach the mixing tip to the top of the dual-syringe of the flash Chairside Light Whitening Gel. Verify the flow of the gel on gauze or mixing pad, prior to the intraoral application.
- Apply the gel directly to the surface of the teeth. (To whiten single teeth, flash Gingiva Protector and the flash Chairside Light Whitening Gel can also be applied lingual). The gel should be applied about 1 - 2 mm thick (avoid contact with other parts of the body and clothes).
- Place the flash Whitening Lamp close to the patient's mouth during the treatment. Turn on the lamp. Eye protection with safety goggles for patient/operator are mandatory.
- Allow the gel to work 15 minutes. Reduce the application time and cycles if patient experiences sensitivity.
- Remove the gel from the teeth using suction - do not rinse - then apply a fresh layer of gel to the teeth. This process can be repeated three times.

Chairside Light Whitening System

50

- After the last treatment remove the gel - using suction - then rinse it off with water and suction. Now the flash Gingiva Protector, flash lip retractor and cotton rolls can be removed. Finally the patient should rinse his/her mouth with warm water.

**The tooth color after the treatment should be documented. Finish off the treatment and lower the risk of unpleasant sensitivity for the patient with flash After Whitening Mousse: apply to the teeth and leave on for about 10 minutes. Recommend flash Whitening Toothpaste with 30 % Xylitol and 15 % hydroxylapatite to your patient to maintain the whitening results and to protect the teeth.**

#### Internal bleaching of endodontically treated teeth:

Wait at least one week following the endodontic procedure before performing a non-vital tooth whitening and ensure that the tooth is endodontically sound.

1. Check the X-rays, to ensure that an adequate apical endodontic sealer is in place and to check where the alveolar bone margin runs.
2. Remove the filling material from the access cavity until the depth you have defined on basis of the X-ray image. Continue to remove the filling material and / or temporary endodontic materials in order to allow the tooth whitening agent sufficient contact with the internal structure of the tooth, to enable for optimal results.
3. Seal the root filling with conventional glassionomer cement. Apply a 1-2 mm thick layer to ensure a secure seal.

4. The internal whitening is performed according to the instructions for vital teeth. Cover the gingiva around the tooth with flash Gingiva Protector. Apply flash Chairside Light Whitening Gel into the cavity and make sure the tooth structure is covered with 1-2mm gel. Apply the gel also on the outside surface of the tooth. Use the flash whitening lamp to activate the gel on the outside tooth surface. The light should not be directed directly into the cavity. Wait 15-20 minutes. Suck of the gel. Procedure might be repeated for 1 or 2 times.

5. Use non-eugenol temporary restorative material to seal the cavity.
6. Before starting to work on adjacent teeth, it is recommended to wait at least two weeks after the whitening treatment - in order to allow the tooth color to stabilize.

#### Side effects

In some cases there may be sensitivities and irritation of the teeth with more or less severe pain during or after treatment. These usually disappear within a short period of time. In case of excessive sensitivity of the teeth during the treatment, the application time of the whitening gel should be shortened or the treatment should be discontinued.

However, to achieve an acceptable whitening result, a split of the treatment in several sessions can be performed in such cases. Sensitivity of the teeth can be reduced with a subsequent application of flash After Whitening Mousse.

Some studies point to a temporary change in the organic matrix of teeth and a reduction in micro hardness of enamel and dentine, but without pathological relevance.

Endodontic bleaching on an imperfect and unexecuted root canal filling not until or beyond the cemento-enamel junction may result in damage to the periodontium (external root resorption) with loss of the tooth. It is essential to draw up the gingiva protection on the tooth neck (about 0,5 -1mm, rule "no pink"). Not sealed gingival protection may cause irritation of the gingiva through contact with peroxide. In this case, the bleaching treatment must be stopped and re-prepared after thorough rinsing and drying of the affected area.

#### Contraindications

WHITeSmile fläsh Chairside Light Whitening System should not be used in the following cases: stains which can be removed by suitable polishing agents and rotating brushes during a professional tooth cleaning, pregnancy and lactation due to a lack of studies, known allergies to any of the ingredients, exposed dentin, insufficient fillings, unmet primary and secondary caries, very heavy smokers and alcoholics, 2 weeks or less before an upcoming dental treatment due to the lack of adhesion of composite fillings and brackets, persons under the age of 18, under anaesthesia.

#### Interactions

Hydrogen peroxide weakens the polymerization of polymers. Therefore, composite fillings or bracket bonds should be performed not earlier than two weeks after a whitening treatment, due to a lack of adhesion.

#### After Whitening Treatment

For at least 4 hours after whitening session, the patient should avoid consuming tea, coffee, red wine, cigarettes etc. as this will reduce effectiveness of treatment.

#### Storage and shelf life

The Gel syringes are packed separately for easier storage in the refrigerator. fläsh Chairside Light Whitening Gel needs to be stored in a cool place or refrigerator (4 - 12 °C). The rest of the kit should not be stored above 25 °C. Do not freeze the gel and protect it from heat and sunlight. The expiry date on the package or syringe refers only to products that have been stored correctly.

#### Ingredients of the WHITeSmile fläsh Chairside Light Whitening Gel

Aqua, Hydrogen Peroxide, Silica, Glycerol, Organic amines, Polyglycols, chlorophyll.

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89

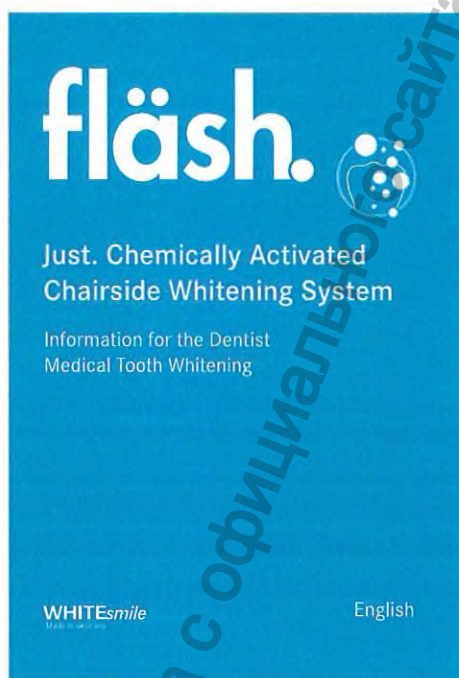
# fläsh.

**Just. Chemically Activated  
Chairside Whitening System**

**Kits for use as touch-up  
whitening procedure without  
the fläsh Whitening lamp.**


**Instructions on the following  
pages =>**

54

**fläsh Just 1 patient kit (REF 3501R):**

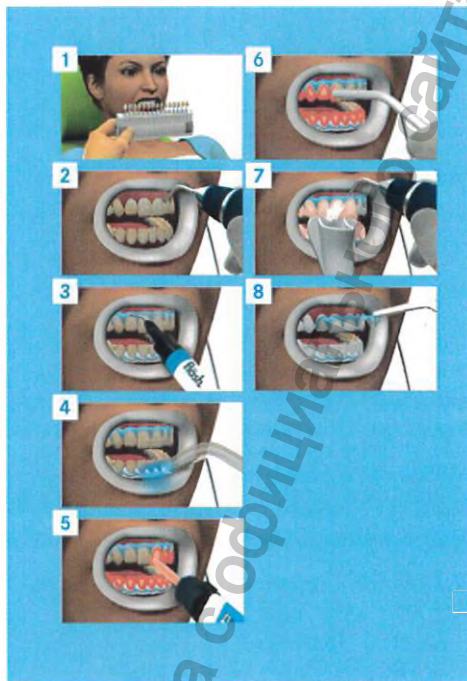- 2,5 ml Gel 38% HP (mixed 30%)
- 1,5 g Gingiva Protector
- AWM 1,2 ml
- Tips 4 pcs, shadeguide

**WHITEmile fläsh Just Chemically Activated Chairside Whitening System**

**Medical Tooth Whitening System for chairside use in the dental clinic for discolored vital and non-vital teeth, only with medical indications. Warning: Keep out of the reach of children. According to the Medical Device Directive 93/42/EEC.**

**IMPORTANT NOTE:** fläsh Just Chemically Activated Whitening System is unique in technique and formulation. Please carefully read and understand the instructions before proceeding with the whitening treatment.

fläsh Just Whitening Gel is a 2-component material for whitening discolored teeth. It is intended only for professional medical whitening in the dental office. By mixing the two components of the bleaching agent the gel gets activated. The duration of the treatment is determined individually and depends on the sensitivity of the teeth, to the desired degree of whitening and the nature and extent of the tooth discolorations.

fläsh Just Whitening Gel is available in 38% hydrogen peroxide (mixed 30%) The red colored gel can be applied directly from the mixing tip to the teeth - its viscous consistency prevents accidental dripping or flow. Unused gel can be kept in the closed syringe until the end of the shelf-life. It can be used with a new mixing tip for another treatment.

ENGLISH

Just, Chemically Activated Chairside Whitening System

56

#### Indication for use

fläsh Just Whitening System is used for fast, medically indicated whitening of strongly discolored, vital and non-vital teeth, which cannot be converted by a professional dental tooth cleaning in the natural or normal tooth color. Medical tooth whitening is indicated when discolorations lead to a deterioration of (health-related) quality of life of the patient because of their abnormally deviating tooth color. Medical tooth whitening can also be used as an alternative to (significantly more invasive) conservative treatment of the affected teeth with veneers or crowns.

Natural or normal are tooth colors as they usually occur in their brightness, translucency and color in the population.

For diagnosis and indication for example the A1-D4 Vita classical shade guide can be used (for the most frequently occurring tooth colors).

Examples of the causes of such (intrinsic) discoloration are: tetracycline/minocyclin discolorations, moderate to severe dental fluorosis (TF index > 2), non-vital teeth (pulp necrosis) - e.g. by trauma, genetic or age-related (degenerative) discoloration, porphyria, or jaundice, heavy deposits of foods with penetration into the tooth substance (intrinsic discoloration by extrinsic dyes).

It can be used to whiten single or multiple teeth or parts of teeth. It can also be used for whitening of single endodontically treated non-vital teeth including intracoronal bleaching.

#### Warning

Because of the high concentration of Hydrogen Peroxide fläsh Just Whitening Gel is highly reactive. To avoid contact with skin or eyes we recommend the use of gloves and safety goggles. For safety reasons, the patient should also wear safety goggles. If the gel does come into contact with skin, remove it and rinse off with water immediately. White spots may appear in the affected area and the patient may experience a burning sensation.

The symptoms should disappear after approx. 30 - 60 minutes. If the gel comes into contact with the eyes, rinse immediately and consult a doctor.

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### 1) Preparation

- **Before the whitening treatment a diagnostic indication for „medical indicated tooth whitening“ has to be defined by the dentist.** After examining the patient the current tooth color should be determined and documented by the dentist. A professional tooth cleaning is indicated prior (ideally 1-2 weeks) to the whitening treatment. Important: due to the fact that restorative materials cannot be whitened, it is recommended that teeth be whitened prior to prosthetic restorations. Any prosthetic treatment (crowns, fillings etc.) should be done 1 - 2 weeks after the whitening treatment in order to match the color of the whitened teeth.
- Start the whitening procedure by applying a thin layer of lip protection on the patient's lips. The teeth should be kept free, by using a lip retractor e.g. the flash lip retractor (not provided in the kit, can be ordered separately), in the patient's mouth. If necessary use cotton rolls.

### 2) Application of flash Gingiva Protector

- flash Gingiva Protector is an easy to apply light-curing resin, which allows optimal covering of soft tissue and neighbouring teeth during the whitening treatment. It is also suitable to cover exposed dentin (e.g. tooth necks) and filling materials during the treatment.
- Attach the dispensing tip to the top of the syringe of the flash Gingiva Protector.

- Dry the gingiva carefully to ensure that the flash Gingiva Protector will hold and seal during the complete treatment. Place the flash Gingiva Protector directly on the gingiva and cover 3 - 4 mm of the gingiva and 0.5 - 1 mm of the teeth (use rule "no pink"). Work in quadrants: light cure the gingiva protector after each half jaw finished. Light cure in an arching movement for about 25 seconds. When finished, use dental mirror to check if the seal is tight and no reddish gingiva is visible. Close open spots again with Gingiva Protector and light cure.  
Instead of the flash Gingiva Protector a rubber dam can also be used.
- The patient should be positioned in a slightly declined position (45° degree angle) during the whitening process to be able to swallow comfortable while closing the teeth with light pressure on the flash lip retractor.

### 3) Whitening Procedure

- The gel should be used at room temperature. Attach the mixing tip to the top of the dual-syringe of the flash Just Whitening Gel. Verify the flow of the gel on gauze or mixing pad, prior to the intraoral application.
- Apply the gel directly to the surface of the teeth. (To whiten single teeth, flash Gingiva Protector and the WHITESmile flash Just Whitening Gel can also be applied lingual). The gel should be applied about 1 - 2 mm thick (avoid contact with other parts of the body and clothes).
- Allow the gel to work 15-20 minutes. Reduce the application time and cycles if patient experiences sensitivity.

• Remove the gel from the teeth using suction – do not rinse – then apply a fresh layer of gel to the teeth. This process can be repeated three times.

• After the last treatment remove the gel - using suction - then rinse it off with water and suction. Now the fläsh Gingiva Protector and the lip retractor, e.g. the fläsh lip retractor (not provided in the kit, can be ordered separately) can be removed. Finally the patient should rinse his/her mouth with warm water.

**The tooth color after the treatment should be documented. Finish off the treatment and lower the risk of unpleasant sensitivity for the patient with fläsh After Whitening Mousse: apply to the teeth and leave on for about 10 minutes. Recommend fläsh Whitening Toothpaste with 30 % Xylitol and 15 % hydroxylapatite to your patient to maintain the whitening results and to protect the teeth.**

#### Internal bleaching of endodontically treated teeth:

Wait at least one week following the endodontic procedure before performing a non-vital tooth whitening and ensure that the tooth is endodontically sound.

1. Check the X-rays, to ensure that an adequate apical endodontic sealer is in place and to check where the alveolar bone margin runs.

2. Remove the filling material from the access cavity until the depth you have defined on basis of the X-ray image. Continue to remove the filling material and / or temporary endodontic materials in order to allow the tooth whitening agent sufficient contact with the internal structure of the tooth, to enable for optimal results.

3. Seal the root filling with conventional glassionomer cement. Apply a 1-2 mm thick layer to ensure a secure seal.

4. The internal whitening is performed according to the instructions for vital teeth. Cover the gingiva around the tooth with fläsh Gingiva Protector. Apply fläsh Just Chemically Activated Whitening Gel into the cavity and make sure the tooth structure is covered with 1-2mm gel. Apply the gel also on the outside surface of the tooth. Wait 15-20 minutes. Suck of the gel. Procedure might be repeated for 1 or 2 times.

5. Use non-eugenol temporary restorative material to seal the cavity.

6. Before starting to work on adjacent teeth, it is recommended to wait at least two weeks after the whitening treatment - in order to allow the tooth color to stabilize.

#### Side effects

In some cases there may be sensitivities and irritation of the teeth with more or less severe pain during or after treatment. These usually disappear within a short period of time. In case of excessive sensitivity of the teeth during the treatment, the application time of the whitening gel should be shortened or the treatment should be discontinued. However, to achieve an acceptable whitening result, a split of the treatment in several sessions can be performed in such cases. Sensitivity of the teeth can be reduced with a subsequent application of fläsh After Whitening Mousse.

Some studies point to a temporary change in the organic matrix of teeth and a reduction in micro hardness of enamel and dentine, but without pathological relevance.

Endodontic bleaching on an imperfect and unexecuted root canal filling not until or beyond the cemento-enamel junction may result in damage to the periodontium (external root resorption) with loss of the tooth.

It is essential to draw up the gingiva protection on the tooth neck (about 0,5-1mm, rule "no pink"). Not sealed gingival protection may cause irritation of the gingiva through contact with peroxide. In this case, the bleaching treatment must be stopped and re-prepared after thorough rinsing and drying of the affected area.

#### Contraindications

flāsh Just Whitening System should not be used in the following cases: stains which can be removed by suitable polishing agents and rotating brushes during a professional tooth cleaning, pregnancy and lactation due to a lack of studies, known allergies to any of the ingredients, exposed dentin, insufficient fillings, unmet primary and secondary caries, very heavy smokers and alcoholics, 2 weeks or less before an upcoming dental treatment due to the lack of adhesion of composite fillings and brackets, persons under the age of 18, under anaesthesia.

#### Interactions

Hydrogen peroxide weakens the polymerization of polymers. Therefore, composite fillings or bracket bonds should be performed not earlier than two weeks after a whitening treatment, due to a lack of adhesion.

#### After Whitening Treatment

For at least 4 hours after whitening session, the patient should avoid consuming tea, coffee, red wine, cigarettes etc. as this will reduce effectiveness of treatment.

#### Storage and shelf life

flāsh Just Whitening Gel needs to be stored in a cool place or refrigerator (4 - 12 °C). The rest of the kit should not be stored above 25°C. Do not freeze the gel and protect it from heat and sunlight. The expiry date on the package or syringe refers only to products that have been stored correctly.

#### Ingredients of the flāsh Just Whitening Gel

Aqua, Hydrogen Peroxide, Silica, Glycerol, Organic amines, Polyglycols, red food color.

Just. Chemically Activated Chairside Whitening System

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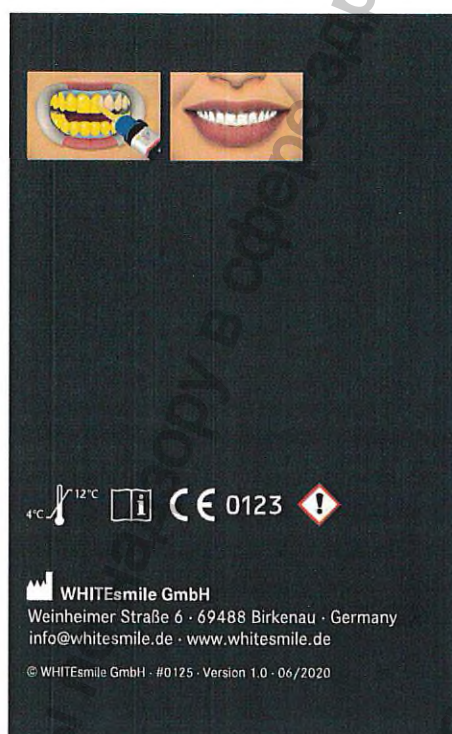
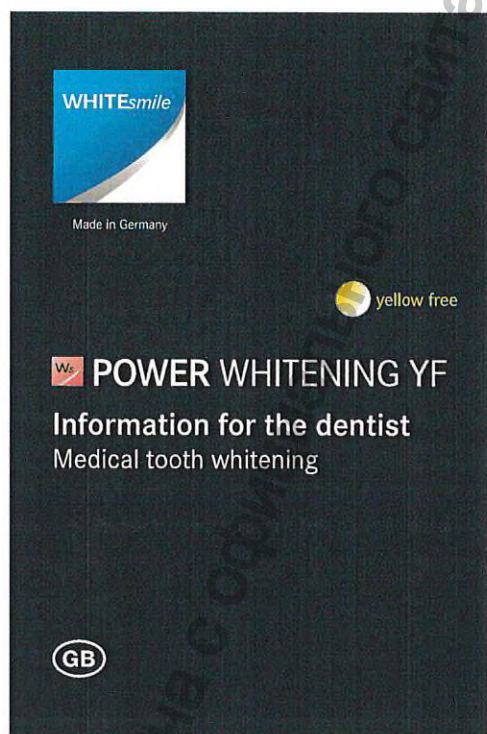
[www.goszdravnadzor.ru](http://www.goszdravnadzor.ru)

## Power Whitening YF

Kits for use as touch-up whitening procedure without the flash Whitening lamp.

Instructions on the following pages =>

**1 Patient kit (REF 5711R):**

- 2,5 ml Gel 40% HP (mixed 32%)
- 1,5 g Gingiva Protector
- AWM 1,2 ml
- tips - 4 pcs
- shadeguide

Power Whitening YF

64

Medical Tooth Whitening System for chair side use in the dental clinic for discoloured vital and non-vital teeth, only with medical indications.

Keep refrigerated 4°C – 12°C. Do not freeze. Warning: Keep out of the reach of a children. According to the Medical Device Directive 93/42/EEC.

#### WHITeSmile® POWER WHITENING YF

Power Whitening YF is a 2-component material for whitening discoloured teeth and contains 40 % hydrogen peroxide prior to mixing (H<sub>2</sub>O<sub>2</sub> - mixed 32 %). It is intended only for professional medical whitening in the dental office. By mixing the two components of the bleaching agent is activated. The duration of treatment is determined individually and depends on the sensitivity of the teeth to the desired degree of whitening and the nature and extent of tooth discolorations. Chemically activated - no light activation needed.

The yellow coloured gel can be applied directly from the mixing tip to the teeth - its viscous consistency prevents accidental dripping or flow. Unused gel can be kept until the end of the shelf-life in the closed syringe. With a new mixing tip it can be used for another treatment.

#### Indication for use

Power Whitening YF is used for fast, medically indicated whitening of strongly discoloured, vital and non-vital teeth, which cannot be converted by a professional dental cleaning in the natural or normal tooth colour. Medically tooth whitening's are indicated when discolorations lead to a deterioration of (health-related) quality of life of the patient because of their abnormally deviating tooth colour. Medical tooth whitening can also be used as an alternative to (significantly more invasive) conservative treatment of the affected teeth with veneers or crowns.

Natural or normal are tooth colours as they usually occur in their brightness, translucency and colour in the population. For diagnosis and indication for example the A1-D4 Vita classical shade guide can be used (for the most frequently occurring tooth colours).

Examples of the causes of such (intrinsic) discolouration are: tetracycline/minocyclin discolourations, moderate to severe dental fluorosis (TF index > 2), non-vital teeth (pulp necrosis) - eg. by trauma, genetic or age-related (degenerative) discolouration, porphyria, or jaundice, heavy deposits of foods with penetration into the tooth substance (intrinsic discolouration by extrinsic dyes).

It can be used to whiten single or multiple teeth or parts of teeth. It can also be used for whitening of single endodontically treated non-vital teeth including intracoronal bleaching.

#### Warning

Because of the high concentration of Hydrogen Peroxide WHITESmile POWER WHITENING YF Gel is highly reactive. To avoid contact with skin or eyes we recommend the use of gloves and safety goggles. For safety reasons, the patient should also wear safety goggles. If the gel does come into contact with skin, remove it and rinse immediately with water. White spots may appear in the affected area and you may experience a burning sensation. The symptoms should disappear after approx. 30 – 60 minutes. If the gel comes into contact with the eyes, rinse immediately and consult a doctor.

#### 1) Preparation

- **Before the bleaching treatment a diagnostic indication for "medical indicated tooth whitening" has to be defined by the dentist.** After examining the patient the current tooth colour should be documented. It is advisable to begin the treatment with a professional tooth cleaning. The teeth should be kept free by using a lip holder, lip retractor or similar instrument. If necessary, use cotton wool balls and apply a lip care to the patient's lips. Because restorative materials will not whiten, it is recommended that teeth are whitened prior to placing aesthetic restorations. Any dental work (crowns, fillings etc.) should be done 1 – 2 weeks after the treatment in order to match the colour of the newly whitened teeth.

#### 2) Application of GINGIVA PROTECTOR

- GINGIVA PROTECTOR is an easy to apply light-curing resin which allows optimal covering of soft tissue and neighbouring teeth during whitening. It is also suitable to cover exposed tooth necks and filling materials during the treatment.
- Attach the dispensing tip to the top of the syringe of the GINGIVA PROTECTOR.

- Dry the gingiva carefully to ensure that the GINGIVA PROTECTOR will hold and seal during the complete treatment. Place the GINGIVA PROTECTOR directly on the gingiva and cover 3 – 4 mm of the gingiva and 0.5 – 1 mm of the teeth. Work in quadrants: light cure the gingiva protector after each half jaw finished. Light cure in an arching movement for about 25 seconds. When finished, use dental mirror to check if the seal is tight and no reddish gingiva is visible. Close open spots again with Gingiva Protector and light cure.
- Instead of the Gingiva Protector a rubber dam can also be used.

### 3) Whitening procedure

- The gel should be used at room temperature. Attach the mixing tip to the top of the dual-syringe of the WHITESmile POWER WHITENING YF Gel. On gauze or mixing pad, verify the flow of the gel prior to applying intraorally.
- Apply the gel directly to the surface of the teeth (to whiten single teeth, GINGIVA PROTECTOR and the WHITESmile POWER WHITENING YF Gel can also be applied lingual). The gel should be applied about 1 – 2 mm thick (avoid contact with other parts of the body and clothes).
- Allow the gel to work about 15-20 minutes. Reduce the application time and cycles if patient experiences sensitivity.
- Remove the gel from the teeth using suction – do not rinse – then apply a fresh layer of gel to the teeth. This process can be repeated three times. (3 x 15-20 min)
- After the last treatment remove the gel first using suction and then by rinsing it off by water.
- Now the GINGIVA PROTECTOR, lip holder and cotton wool balls can be removed. Finally the patient should rinse their mouth with warm water.

The tooth colour after treatment should be documented for your records. You can finish off the treatment and lower the risk of unpleasant sensitivity for the patient with AFTER WHITENING MOUSSE – apply to the teeth and leave on for about 10 minutes. Recommend WHITESmile WHITENING TOOTHPASTE with 30 % Xylitol to your patient to maintain the results and to protect the teeth.

#### Internal bleaching of endodontically treated teeth:

Wait at least one week following the endodontic procedure before performing a non-vital tooth whitening and ensure that the tooth is endodontically sound.

1. Check the X-rays, to ensure that an adequate apical endodontic sealer is in place and to check where the alveolar bone margin runs.
2. Remove the filling material from the access hole until the depth that you have defined on basis of the X-ray image. Continue to remove the filling material and / or temporary endodontic materials in order to allow the tooth whitening agent contact with the internal structure of the tooth. Only this procedure allows best results.
3. Seal the root with conventional glass ionomer cement. Apply a 1-2 mm thick layer to ensure a secure seal.
4. The internal whitening is performed according to the instructions for vital teeth. Cover the gingiva around the tooth with Gingiva Protector. Apply POWER WHITENING YF gel into the cavity and make sure the tooth structure is covered with 1-2mm gel. Apply the gel also on the outside surface of the tooth. Wait 15-20 minutes. Suck of the gel. Procedure might be repeated for 1 or 2 times. After the last application suck of gel and rinse off remaining gel from cavity.

Power Whitening YF

66

5. Use non-eugenol temporary restorative material in opening.

6. If result is not sufficient you might use WHITEsmile 35 FICE Gel for walking bleaching technique.

7. Before starting to work on adjacent teeth, it is recommended to wait at least two weeks after the whitening treatment - stabilise the tooth colour

#### Side effects

In some cases there may be sensitivities and irritation with more or less severe pain during or after treatment. It usually disappears within a short period of time. In case of sensitivity of the teeth during the treatment, the application of whitening gel should be shortened or the treatment should be discontinued. However, to achieve acceptable whitening results, the treatment in several sessions should be performed. Sensitivity of the teeth can be reduced with the application of AFTER WHITENING MOUSSE.

Endodontic bleaching on an imperfect and unexecuted root canal filling not until or beyond the cemento-enamel junction may result in damage to the periodontium (external root resorption) with loss of the tooth.

The gum protection is essential to pull up on the tooth (0,5 -1mm). Not sealed gingival protection may cause the gums through contact with peroxide. In this case, the treatment must be stopped and re-prepared after thorough drying the tooth.

#### After Whitening Treatment

For at least 4 hours after whitening session, the patient should avoid consuming tea, coffee, red wine, cigarettes etc. as this will reduce effectiveness of treatment.

#### Contraindications

Power Whitening YF should not be used in the following cases: stains which can be removed by suitable polishing agents and rotating brush during a professional tooth cleaning, Pregnancy and lactation due to lack of studies, known allergy to any of the ingredients, exposed dentin, faulty edges of fillings, unmet primary and secondary caries, very heavy smokers and alcoholics, 2 weeks or less before an upcoming dental treatment due to the lack of adhesion of composite fillings and brackets, persons under the age of 18, under anaesthesia.

#### Interactions

Hydrogen peroxide weakens the polymerization of polymers. Therefore, composite fillings or bracket bonds should be performed not earlier than two weeks after a whitening treatment, due to lack of adhesion.

#### Storage and shelf life

WHITEsmile POWER WHITENING YF Gel should be stored in a cool place or refrigerator (4 - 12 °C). The expiry date on the package or syringe refers only to products that have been stored correctly. Do not freeze the gel and protect it from heat and sunlight.

#### Ingredients

Aqua, Hydrogen Peroxide, Silica, Glycerol, Organic amines, Polyglycols, yellow food colour.

## Signs and symbols materials

CE Mark class 1 medical device in EU

CE Mark class IIa medical device in EU with number NB

Manufacturer

Storage temperatur 4-12°C

Storage temperatur 4-25°C

Read user manual cosmetic

expiry date

Article number

Caution

LOT number

Read user manual medical device

## Storage and Transport of materials

### Storage

Storage for REF 3103R/3106R: according to the labelling on the boxes (4-25 °C), dry, cool, dust free and hermetically locked, do not expose the product to heat or direct sunshine.

Storage for Whitening gels as part of Kit 3103R/3106R, REF 3501 / 5711: according to the labelling on the boxes, 4-12 °C, dry, cool, dust free and hermetically locked, do not expose the product to heat or direct sunshine.

### Transport

All WHITEsmile Gels are safe to be transported without temperature control for a period of up to 10 days. Temperature peaks of up to 50°C are not causing any change to the product. This procedure is in place since the year 2008 without any reported damage. It is only necessary to keep the product at storage in the limits given on the boxes. Dur to shipment in all hot countries with this condition since more than 10 years the procedure is considered safe.

In addition, a damage would be clearly visible, because the gel would dry out and a use would not be possible anymore.

Humidity for transport, storage and use for kits: 5% to 95%

## Disposal of fläsh Whitening Lamp

The expected service life of the fläsh whitening lamp is 10 years.

Based on Directive 2012/19/EU and country-specific disposal directives for waste electrical and electronic equipment, we would like to point out that these must be disposed of in a special way within the European Union (EU). These regulations require environmentally compliant recycling/disposal of waste electrical and electronic equipment. They must not be disposed of as domestic waste. This is shown with the symbol of the crossed out dust bin, which has been in use since March 24, 2006.

For disposal procedure please contact your local dealer.

### Other countries

The dental dealers would be glad to provide you with country-specific information. Not dangerous waste, disposal in accordance with Russian legislation, in accordance with the law regulations at the time of disposal.

## Disposal of materials

Must be sent for special disposal in accordance with regulations on special waste. Disposal in accordance with Russian legislation, in accordance with the law regulations at the time of disposal. Dismantling and disposal is carried out under a separate contract.

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[www.gosdrazhnadzor.ru](http://www.gosdrazhnadzor.ru)

# fläsh.

## General instructions for fläsh Professional Tooth Whitening System

Not containing drug.  
Representative company in Russia:  
OOO Stomatorg, 117485, Moscow, st. Profsoyuznaya 88/20, floor a1, quarters I, room 35,  
8 (495) 620-97-34, mail@stomatorg.ru

**WHITEsmile GmbH**  
Weinheimer Straße 6  
69488 Birkenau  
Germany

**flaesh.com**

Phone: +49 (0) 62 01 / 8 43 21-90  
Fax: +49 (0) 62 01 / 8 43 21-99  
E-Mail: info@whitesmile.com

© WHITEsmile 2021  
Subject to technical changes  
and errors in the text.  
Version 1.1 · 06/2021 Date  
02.06.2020  
fläsh Whitening Lamp is  
Made in Germany

Notare Dr. Ott & Dr. Zander | Ehretstr. 14 | 69469 Weinheim  
Telefon 06201 / 84522 - 0 | Telefax 06201 / 84522 - 22  
E-Mail kanzlei@notare-weinheim.de | Internet www.notare-ott-zander.de

### Certification of signature

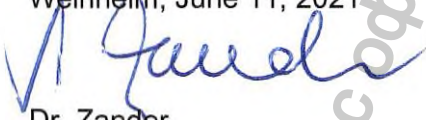
I, notary, hereby certify, that the above is the true signature, acknowledged in my presence, of

Mr. Benno Götz Walter,  
born 07 May 1966,  
residing at 69488 Birkenau, Weinheimer Str. 6,

-identified by identity card-

The certification refers only to the signature and not to the content of the declarations of the undersigning person.

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