ASA DENTAL

MAKE PEOPLE SMILE

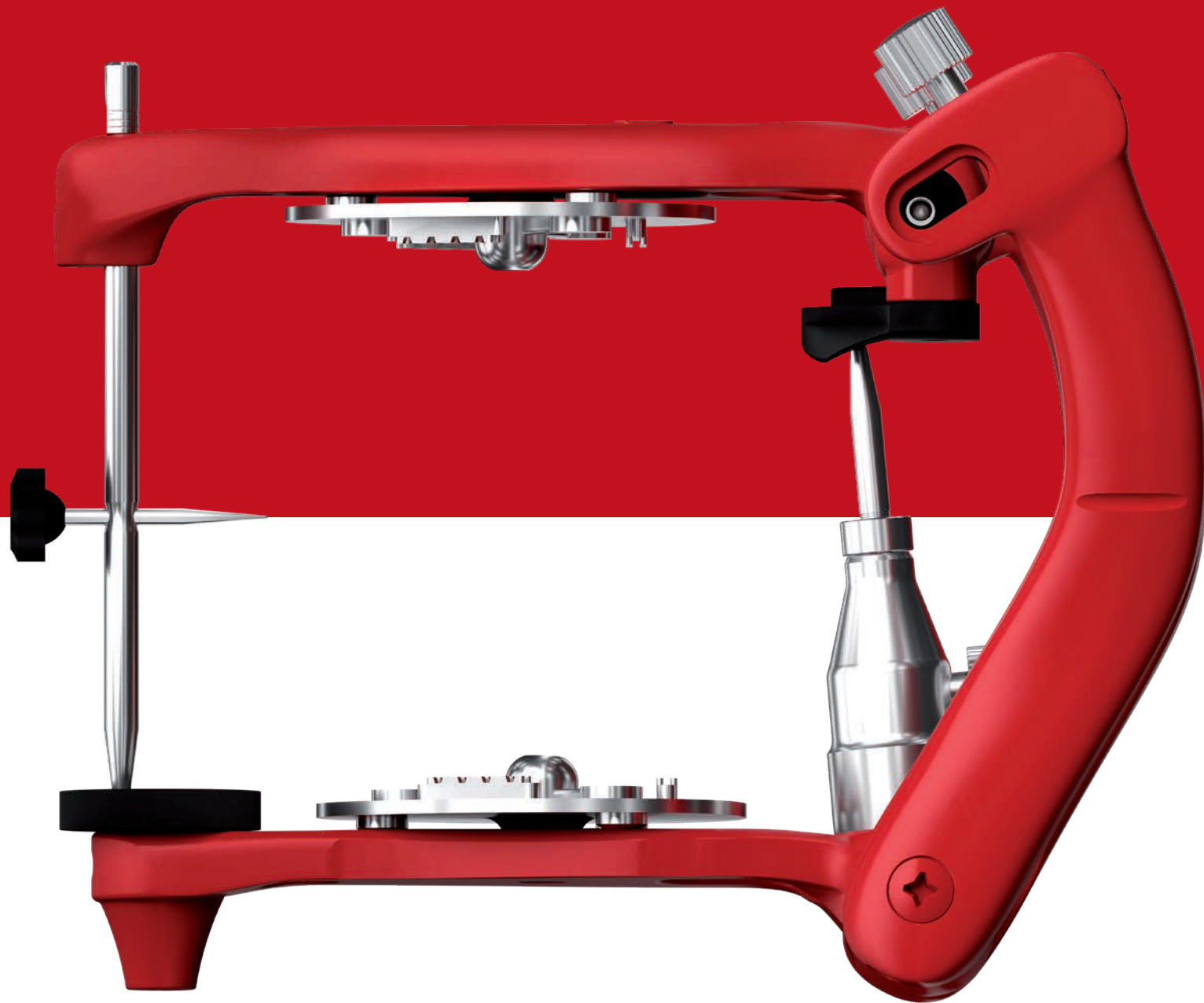
CTOMAPT
DENTAL SOLUTIONS

MADE IN ITALY

INSTRUCTIONS FOR USE

ISTRUZIONI D'USO

5050
5050Plus

PRODUCT DESCRIPTION

5050 and 5050Plus are non-Arcon average value articulators designed to help dental technicians meet dentists' requests for high precision, regardless of the prosthetic technique. A light, stable and ergonomic construction facilitates set-up and work on any models. Additional features guarantee an exact simulation of the condylar path with a high fitting accuracy, giving prosthesis and restorations the necessary occlusion and function throughout the full range of mandibular movements.

INDICATIONS


The articulator is suitable for the fixation of models for lab-fabricated reconstructions. 5050 and 5050Plus allow a perfect simulation of average movement patterns of the human temporomandibular joint (TMJ).

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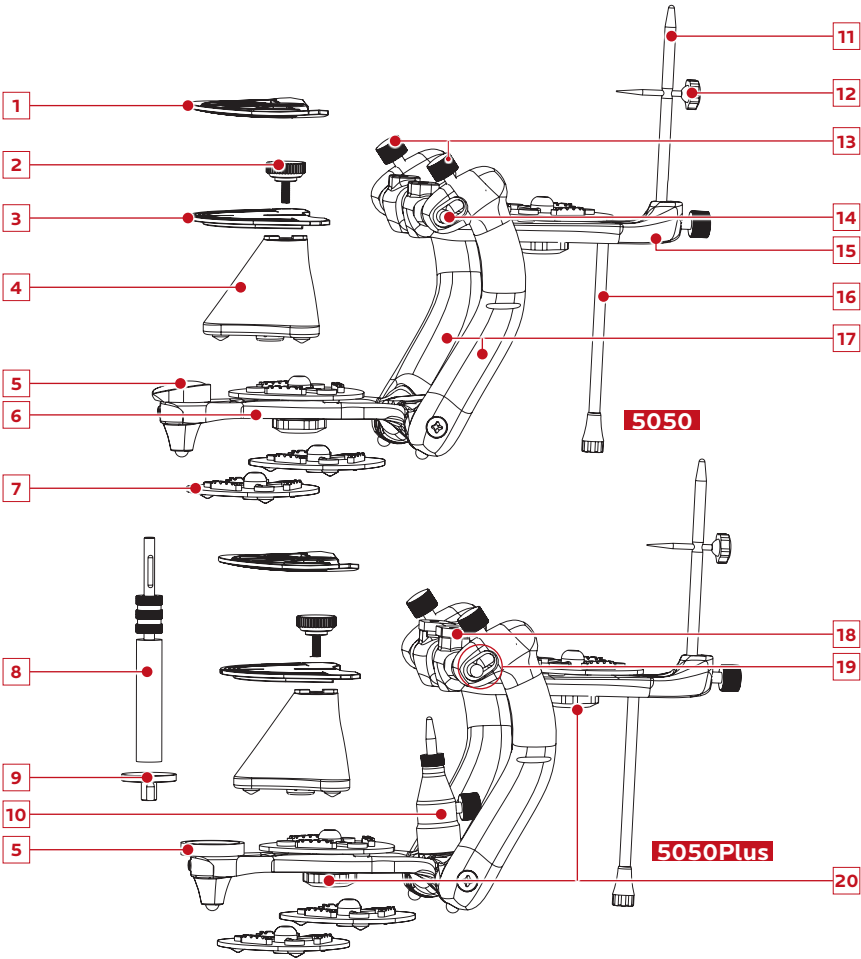UNPACKING AND CHECKING THE CONTENTS

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5050Plus

ASSEMBLY AND INITIAL SETUP

Mount above parts depending on the selected technique or reconstruction stage.

Articulators 5050 and 5050Plus can conveniently work in 4 different positions
(all these positions allow a superb overview of the model mounted in the articulator):

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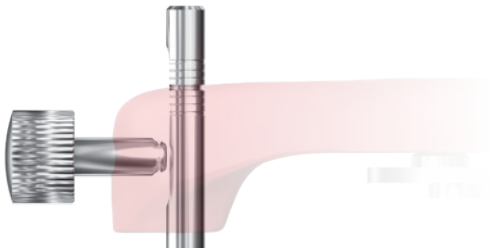
The anti-rotational system for the incisal rod ensures that positioning is always correct and accurate.

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4 elliptic references help the technician regardless the arch size and the selected starting technique.

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The back rod system allow to remove the incisal rod without affecting performances.

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COMPARISON

Technical features	5050	5050Plus
stable 60° operation for technician comfort	✔	✔
ideal for class III patients	✔	✔
compatible with any pressure polymerizers	✔	✔
anti-rotation system for incisal rod	✔	✔
template with elliptic references for lower arches models	✔	✔
occlusal table with elliptic references for upper arches models	✔	✔
magnetic rod kit	○	✔
back rod system	○	✔
4 mounting plates	✔	✔

Technical data	5050	5050Plus
intercondylar distance	110 mm	110 mm
condylar inclination fixed at	30°	30°
Bennett angle	0 - 7°	0 - 7°
incisal table inclination	10°	0°
inclined contrast table	-	24°
distance between mounting plates	87 mm	87 mm
opening	180°	180°

MAINTENANCE AND CLEANING

Monitoring and maintenance		
CHECK	PART	FREQUENCY
joint sufficiently lubricated	joint bolt	monthly/as necessary
metallic knob is dirty	metallic knob for template/occlusal table	weekly/as necessary

Cleaning*		
CHECK	FREQUENCY	MEASURE
metallic knob contaminated with wax or plaster	weekly/as necessary	rinse with warm water
joints contaminated with dust	weekly/as necessary	rinse with warm water

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105 Maxess Road, Suite 124
 Melville, New York 11747 - USA

Asa Dental CHINA CO. LTD.

Room 315, Huaying Building,
 Zhong Xin Avenue
 Tianjin Airport Economic Area
 Tianjin - China

Asa Dental S.p.A. (Branch)

Dubai South Business Center
 Building C, 3rd Floor
 Dubai Logistics City
 PO Box 390667
 Dubai - UAE

ASA DENTAL

MAKE PEOPLE SMILE

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Asa Dental USA Corp.