Philips Zoom! WhiteSpeed Led Accelerator

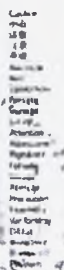
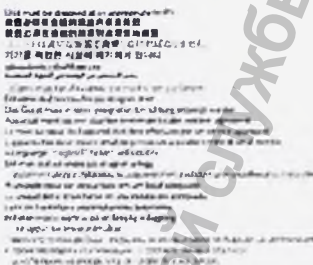
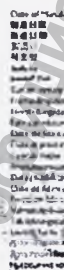
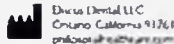
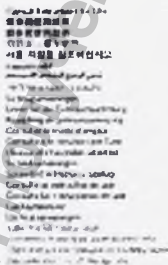
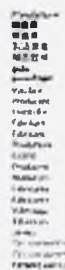
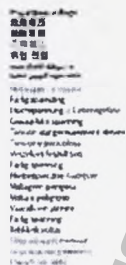
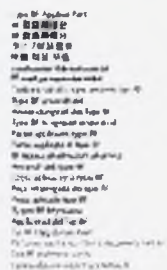
Directions for Use

(cover page only, not part of actual DFU)

Authorized by:
Kristian Nielsen
Discus Dental LLC/Philips Oral Healthcare
Regulatory Affairs Team Leader

Signature: _____

SN

Whitening LED Accelerator Whitening	
美白LED 加速器 歯 whitening	10
美白 LED 加速器 歯 whitening	10
ホワイトニングLEDアクセラレータ 日本編	27
화이트닝 LED 액셀러레이터 한국어	21
Whitening Accelerator (LED) Whitening	3
Whitening Accelerator (LED) Whitening	3
Whitening LED lamp (1st edition) (Whitening) Only	4
LED-accelerator (Whitening) Dental	5
LED-Acceleratorlampe zur beschleunigten Zahn-Whitening Deutsch	5
Whitening LED-vorsteller Nissan lance	6
Accelerateur de blanchiment par LED Français	7
Acceleratore e blanchimento a LED Italiano	7

First world LED growth Mapper	82
Philips (LED) indicator Alora	68
Vibrating LED Accelerator Publi	94
Longer lifetime per lamp type and better protection	100
LED indicator rib (luminescence) Equid	106
Manipulation LED indicator Suvo	112
LED indicator for lighting Sovda	118
Long-life LED indicator Tutice	124
Competition in LED indicator Lamp 2000	130
Competition in LED indicator Lamp 2000	136
Energy saving LED Dunlop	142

Zoom! WhiteSpeed

PHILIPS
ZOOM!

1. Product overview

1.1. Packaging Contents

Each Zoom! WhiteSpeed Whitening LED Accelerator (LED Accelerator) has been thoroughly inspected and tested at the factory. Upon arrival at your facility inspect the shipping container and the device for damage. Any damage should be reported immediately to your Philips representative. In the case of a damaged shipment, please save all packaging materials.

1. LED Accelerator Head (1)
2. LED Accelerator Boom (1)
3. LED Accelerator Stand with Power Pack (1)
4. LED Accelerator Base (1)
5. Casters (3)
6. Protective Eyewear (3)
7. Parts Bag
8. Allen Wrench (1)
9. Screw (1)
10. Bolt with Knob (1)
11. Lock Washer (2)
12. Fender Washer (1)
13. User's Manual (1)
14. Warranty Card (1)
15. Light Guides (2)
16. Power Cord (1)

1.3. Intended Use
The Zoom! WhiteSpeed Whitening LED Accelerator is a chairside teeth whitening light designed to be used by a dental professional for use in whitening discolored teeth.

1.4. Device Description
The LED Accelerator is a chairside teeth whitening light designed to be used by dental professionals for use in whitening discolored teeth. The LED Accelerator's light source has been designed to deliver the optimal spectrum of light energy to activate Zoom! WhiteSpeed Whitening Gel to deliver up to 6-8 shades* of teeth whitening. Wavelength emissions are between 400 to 505 nanometers (nm) and the energy intensity delivered to the teeth can be set at three levels depending on patient and case type. The LED Accelerator must be used at all times in accordance with instructions and precautions outlined within the user's manual.

1.5. Safety Precautions
Do not attempt to use the LED Accelerator without a thorough understanding of this manual and the manual within the Zoom! Light-Activated In-Office Whitening Kit (Kit). Failure to do so may result in injury to you or your patients or damage to the product. Only allow trained and qualified personnel to operate the system.

1.6. LED Accelerator Assembly
Remove all items from shipping carton.
Snap the casters into the feet of the LED Accelerator Base and lock the casters.
Place the LED Accelerator Base upright in the floor insert. The Stand into the Base by aligning the pin in the Stand with the slot in the Base and pushing the Stand securely into place.
Assemble the lock washer with the bolt teeth facing each other and place it on the bolt followed by the fender washer. Reaching under the Base, screw the bolt into the bottom of the Base, securing it to the Stand.
To attach the LED Accelerator Head, loosen the knobs by the socket at the end of the Boom by 1-5 turns. Gently but firmly, push the Head into the socket. It will be held securely in place by the socket springs.
Place the LED Accelerator Boom onto the Stand by carefully aligning the hole in the Boom with the hole in the yoke. Insert this screw into the hole and tighten. Keep tightening the screw until it is completely in the pocket. The screw should be flush or just into the pocket. The screw has glue on the threads (yellow in color) that retracts as the screw is tightened. Therefore, it is important to finish tightening the screw upon insertion or the glue may stop the screw from being fully inserted.
Connect the electrical cord from the Boom to the cord from the Stand and the Head. The head connector must be turned until you feel it lock into place.

1.7. Authorized representatives in the territory of Russian Federation
All questions related to system operation, service, quality etc. in the territory of Russian Federation should be addressed to:
Philips LLC
13, Sergeeva Makreev St., Moscow 123022, Russia
Tel/Fax: +7 (495) 937 93 00/8 800 300 09 80

1.8. Based on the Viascan Clinical shade guide arranged in value order, Vita and Vitaspan are trademarks of Vita Zahnfabrik AG, Germany

1.3. Safety Precautions
Do not attempt to use the LED Accelerator without a thorough understanding of this manual and the manual within the Zoom! Light-Activated In-Office Whitening Kit (Kit). Failure to do so may result in injury to you or your patients or damage to the product. Only allow trained and qualified personnel to operate the system.

1.6. LED Accelerator Assembly

1. Remove all items from shipping carton.
2. Snap the casters into the feet of the LED Accelerator Base and lock the casters.
3. Place the LED Accelerator Base upright in the floor insert. The Stand into the Base by aligning the pin in the Stand with the slot in the Base and pushing the Stand securely into place.
4. Assemble the lock washer with the bolt teeth facing each other and place it on the bolt followed by the fender washer. Reaching under the Base, screw the bolt into the bottom of the Base, securing it to the Stand.
5. To attach the LED Accelerator Head, loosen the knobs by the socket at the end of the Boom by 1-5 turns. Gently but firmly, push the Head into the socket. It will be held securely in place by the socket springs.
6. Place the LED Accelerator Boom onto the Stand by carefully aligning the hole in the Boom with the hole in the yoke. Insert this screw into the hole and tighten. Keep tightening the screw until it is completely in the pocket. The screw should be flush or just into the pocket. The screw has glue on the threads (yellow in color) that retracts as the screw is tightened. Therefore, it is important to finish tightening the screw upon insertion or the glue may stop the screw from being fully inserted.
7. Connect the electrical cord from the Boom to the cord from the Stand and the Head. The head connector must be turned until you feel it lock into place.

1.3. Intended Use

The Zoom! WhiteSpeed Whitening LED Accelerator is a chairside teeth whitening light designed to be used by a dental professional for use in whitening discolored teeth.

1.4. Device Description
The LED Accelerator is a chairside teeth whitening light designed to be used by dental professionals for use in whitening discolored teeth. The LED Accelerator's light source has been designed to deliver the optimal spectrum of light energy to activate Zoom! WhiteSpeed Whitening Gel to deliver up to 6-8 shades* of teeth whitening. Wavelength emissions are between 400 to 505 nanometers (nm) and the energy intensity delivered to the teeth can be set at three levels depending on patient and case type. The LED Accelerator must be used at all times in accordance with instructions and precautions outlined within the user's manual.

1.5. Safety Precautions
Do not attempt to use the LED Accelerator without a thorough understanding of this manual and the manual within the Zoom! Light-Activated In-Office Whitening Kit (Kit). Failure to do so may result in injury to you or your patients or damage to the product. Only allow trained and qualified personnel to operate the system.

1.6. LED Accelerator Assembly
Remove all items from shipping carton.
Snap the casters into the feet of the LED Accelerator Base and lock the casters.
Place the LED Accelerator Base upright in the floor insert. The Stand into the Base by aligning the pin in the Stand with the slot in the Base and pushing the Stand securely into place.
Assemble the lock washer with the bolt teeth facing each other and place it on the bolt followed by the fender washer. Reaching under the Base, screw the bolt into the bottom of the Base, securing it to the Stand.
To attach the LED Accelerator Head, loosen the knobs by the socket at the end of the Boom by 1-5 turns. Gently but firmly, push the Head into the socket. It will be held securely in place by the socket springs.
Place the LED Accelerator Boom onto the Stand by carefully aligning the hole in the Boom with the hole in the yoke. Insert this screw into the hole and tighten. Keep tightening the screw until it is completely in the pocket. The screw should be flush or just into the pocket. The screw has glue on the threads (yellow in color) that retracts as the screw is tightened. Therefore, it is important to finish tightening the screw upon insertion or the glue may stop the screw from being fully inserted.
Connect the electrical cord from the Boom to the cord from the Stand and the Head. The head connector must be turned until you feel it lock into place.

1.7. Authorized representatives in the territory of Russian Federation
All questions related to system operation, service, quality etc. in the territory of Russian Federation should be addressed to:
Philips LLC
13, Sergeeva Makreev St., Moscow 123022, Russia
Tel/Fax: +7 (495) 937 93 00/8 800 300 09 80

1.8. Based on the Viascan Clinical shade guide arranged in value order, Vita and Vitaspan are trademarks of Vita Zahnfabrik AG, Germany

*Based on the Viascan Clinical shade guide arranged in value order, Vita and Vitaspan are trademarks of Vita Zahnfabrik AG, Germany

8. When you are ready to test the LED Accelerator activate the unit by turning on the main power switch on the right side of the Power Pack.

1.7 Set up

Plug the power cord protruding from the Base directly into a properly rated wall outlet.

Caution

Inlet (the end of the cord) must be accessible to disconnect power if a problem occurs.

The LED Accelerator is now ready for use with a Zoom! Light-Activated In-Office Whitening Kit. Please refer to the Zoom! Light-Activated In-Office Whitening Kit manual for further set-up and gel application instructions.

2. Zoom! Light-Activated In-Office Whitening Treatment

2.1 Light Guide

The Light Guide is designed to ensure proper distance and alignment to maintain correct light exposure on the smile zone. Remove Light Guide from Zoom! Light-Activated In-Office Whitening Kit and place on end of Head. Turn on the LED Accelerator using power button located on the side of the Power Pack. When the Light Guide is properly attached, the Power Pack display will read "Guide Status: Guide Attached" followed by "X of X Sessions". Each new Light Guide is loaded with four 15-minute activation sessions.

2.2 Align Head and Light Guide to Retractor

Once whitening gel application is complete (see Zoom! Light-Activated In-Office Whitening Kit manual for instructions) raise back of chair so patient is in a slightly reclined position (45 degree angle) and able to swallow comfortably while maintaining light pressure on bite block. Slide Accelerator into place next to patient. The Boom should be movable, but stable.

To align the LED Accelerator Head:

1. Grasp the Head with one hand and bring it close to the patient's mouth. Use your other hand to lift or lower the Boom as needed to make sure the Head is aimed directly at the patient's teeth.

2. Use both hands to finalize positioning of the Head so that Light Guide and IsoPrep Retractor slots are aligned. Tighten the Head and Boom knobs (4). Lock casters in position. While engaged, the Light Guide and IsoPrep Retractor will allow the Head to follow minor movements of the patient's head to ensure proper alignment.

2.3 Running a whitening procedure cycle

Each Zoom! Light-Activated In-Office Whitening Procedure consists of three 15-minute whitening sessions with a pause between each session to allow the operator to remove and replace the whitening gel. Each Light Guide includes a fourth session for use on patients with shades A3 or darker.

1. Ensure the patient is properly prepared according to the Zoom! Light-Activated In-Office Whitening Kit manual with whitening gel and all protective barriers in place.

2. Press the "Intensity" button to toggle through high, medium, and low light intensity until the desired intensity is selected. This setting can be changed at any time during the whitening procedure.

3. Press the "Start" button to turn on light and activate session timer.

4. Once light is on, a countdown will be displayed. The LED Accelerator will beep once when 3 minutes are remaining and beep three times in the final 3 seconds of the 15-minute session. When time reaches zero, light will turn off and a long beep will sound.

5. Point out to the patient that progress of each session will be tracked by 4 white lights on top of the Head, each indicating one quarter of a session.

6. If a session must be interrupted, press the "Pause" button. The remaining session time will blink on the display. To reactivate the session, press either the "Pause" or "Start" button. **DO NOT PRESS THE "STOP" BUTTON AT ANY TIME DURING THIS PROCESS OR AN ENTIRE 15-MINUTE SESSION WILL BE LOST.**

7. At the end of each 15-minute session, the display will indicate the number of remaining sessions left on the Light Guide. Press "Start" to activate the light for the next session.

8. When all sessions on a Light Guide are used, the display will read "Sessions Complete" for three seconds, followed by "Guide Status - Replace Guide". Detach the Light Guide from IsoPrep Retractor and pull LED Accelerator away from patient.

Tip

- Use High setting for maximum whitening. Low setting for maximum heat and sensitivity management. However, please note that lower light intensity may reduce results.
- After each session, make sure the patient's lips and gums are consistently and thoroughly covered to help prevent any soft tissue exposure to the LED lights and any whitening gel.

3. Care and maintenance

The LED Accelerator should be cleaned and disinfected 1 after each whitening session. Do not spray or pour liquids directly onto the system. Apply cleaners to a soft cloth or paper towel before using. Do not use abrasive cleaners or solvents. The following cleaners are recommended: Sanidish, CanWipes, SoCan, Blek, BioSurf, Cidex and IPA. Follow your cleaning products instructions for use to ensure proper cleaning of the unit.

For optimum performance, ensure that the output window on the Head remains clean and free of particles or streaks.

The LED Accelerator has no user serviceable or user-replaceable parts with the exception of two replaceable fuses that are inside the Power Pack housing.

The LED Accelerator complies with relevant international and national law and standards on EMC (electromagnetic compatibility) for this type of product when used as intended. Such laws and standards define both the permissible electromagnetic emission levels from product and its required immunity to electromagnetic interference from external sources.

Other electronic products exceeding the limit defined in such EMC standards could, under unusual circumstances, affect the operation of the product.

- Medical electrical products need special precautions regarding EMC, and need to be installed and put into service according to EMC information provided in this manual.

- The use of accessories and cables other than those specified may result in increased emission or decreased immunity levels.

- The product should not be used adjacent to other products and that if adjacent use is necessary, it should be observed to verify normal operation.

- To avoid the risk of electrical shock, do not remove any system panels.

- Refer servicing to qualified personnel.

- Never operate the System with a damaged power cord.

Storage

The Boom is designed to fold down to minimize overall LED Accelerator footprint for convenient storage or to move the device. To do so, loosen the Boom knobs and fold the Boom down and in towards the Stand.

Disposal

Philips is concerned about the preservation of the environment. Please dispose of this product properly at the end of its useful life according to local laws and regulations for electronic equipment waste.

Caution: Hazardous voltage

4. Fuse replacement instructions

Caution

1. **CAUTION!** Unplug the LED Accelerator from the outlet.
2. Use a Phillips-head screwdriver to remove the screw at the bottom of the Power Supply housing. Set the screw aside for reassembly.
3. Remove the door from the housing.
4. Pull out the white power cord from its connection and let it hang freely.
5. Look for the fuse label near the upper right-hand side to identify the fuse location. (see figures 1-2)
6. Using a small, thin bladed, flat-head screwdriver pry open the right hand side of the fuse door. Then pry out the red fuse rack. (see figures 2-3)
7. Remove both fuses from the red fuse rack - note their location in the fuse rack. (see figures 4-5)
8. Snap in the new fuses at the top of the fuse rack; they should be closer to the side with the tabs (top). (see figures 4-5)
9. Reinsert the fuse rack into the fuse slot with the tabs first. Push the fuse rack in until it is securely in place. You will not be able to close the fuse door if the fuse rack is not inserted correctly.
10. Close the fuse door.
11. Reinsert the power cord firmly into the housing.
12. Reinsert the screw into the door and tighten with a Phillips-head screwdriver.

5. Trouble Shooting Guide

Problem	Causes	Solution
Display reads "SYSTEM ERROR CALL CUSTOMER SERVICE"	• One of the cables between power/make sure all 3 connectors are plugged in (head, back of pack and LED Accelerator head not plugged in the boom, into the power pack) If error code remains, call customer service	
Display reads "GUIDE STATUS" IMPROPER GUIDE	• New Light guide was first placed on Zoom Advanced Power or older system	Replace guide with a new guide that was not first placed on a Zoom Advanced Power or older system
Display reads "GUIDE STATUS" REPLACE GUIDE before the guide is empty	• One or more sessions was interrupted by the off button • One or more sessions was interrupted by a power outage • One or more sessions was interrupted by light guide disconnection	If One or more sessions is interrupted by any of the 3 causes listed, the last session can not be recovered
Display reads "GUIDE STATUS" ATTACH GUIDE When a guide is attached	• Light guide is not fully seated on the head • Light guide was up on upside down • The pins on the underside of the head are damaged	Put the light guide off and reinstall correctly Place the light guide on the head with the word TOP facing up Call Customer service for a head replacement

6. Specifications

Zoom! WhiteSpeedWhitening LED Accelerator		
Inputratings	100-240V ~ 3A, 50-60 Hz	
Classification	Class 1 Type BF, IPX0, Equipment not suitable for use in the presence of flammable mixtures	
Operatingmode	Continuous	
Operating	Ambienttemperature	10°C ~ 40°C (50°F ~ 104°F)
	Relativehumidity	5% to 95% notincludingcondensation
	Atmosphericpressure	70 kPato 106 kPa
Transportandstorage	Ambienttemperature	-40°C ~ 70°C (-40°F ~ 158°F)
	Relativehumidity	5% to 95% notincludingcondensation
	Atmosphericpressure	50 kPato 106 kPa
Lightsource	LED Array	
Bandwidth	400 ~ 505 nanometersnormal	
Cooling	AirCooled	
Height	136cm ± 53.5 in	
Weight	11 kg / 24 lbs	

Guidance and manufacturer's declaration – electromagnetic emissions

The Zoom! WhiteSpeed Whitening LED Accelerator is intended for use in the electromagnetic environment specified below. The customer or the user should assure that it is used in such an environment.

Emissions	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Zoom! WhiteSpeed Whitening LED Accelerator uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The wireless transmitter in the foot pedal is battery powered only, and does not connect to public mains
RF emissions CISPR 11	Class A	The Zoom! WhiteSpeed Whitening LED Accelerator is suitable for use in all establishments other than domestic and may be used in domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes, provided the following warning is heeded
Powerline harmonics IEC/EN 61000-3-2	Class A	Warning: This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. If any interference or disruption occurs, such as re-orienting or relocating the Zoom! WhiteSpeed Whitening LED Accelerator or shielding the location
Powerline flicker IEC/EN 61000-3-3	Complies	

7. Indications, contraindications and side effects

Indications:

- Age discoloration of teeth (as years go by the color of teeth changes due to discoloration of hard tooth tissues. Besides, in due course the surface pigments can become internal ones.);
- Discoloration of teeth after a trauma or disease (discoloration of hard tooth tissues as a result of blood egress from the pulp and the tooth being colored from within.);
- hypoplasia of the hard tooth tissues at the initial stage
- fluorosis / hypoplasia of the hard tooth tissues (at fluorine overabundance in water resulting in its excess in the body. Premolar teeth and maxilla cutters are the most often affected, molars and mandible cutters - the least so.);
- "Tetracycline" teeth
- hypoplasia of teeth (in case of a mechanical trauma of second teeth rudiments or their involvement in an inflammatory process);
- pulpless teeth having changed color;
- teeth having the Vita scale color of A2 (inclusive) and darker;
- discoloration of teeth owing to bad habits, but bleaching will require preliminary ultrasonic teeth cleaning

Contraindications:

Discus Dental, LLC recommends the following categories of patients to consult their doctor before the teeth bleaching procedure:

- pregnant and lactating mothers;
- Those currently treated for a serious illness or disorder, e.g. immune compromised, AIDS, etc.
- Children under the age of 13 years
- Light sensitive individuals including those on PUVA (Psoralen + UV Radiation) or other photo-chemotherapy
- Those taking any photo-reactive drugs or substances, whether over-the-counter, prescription or homeopathic

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